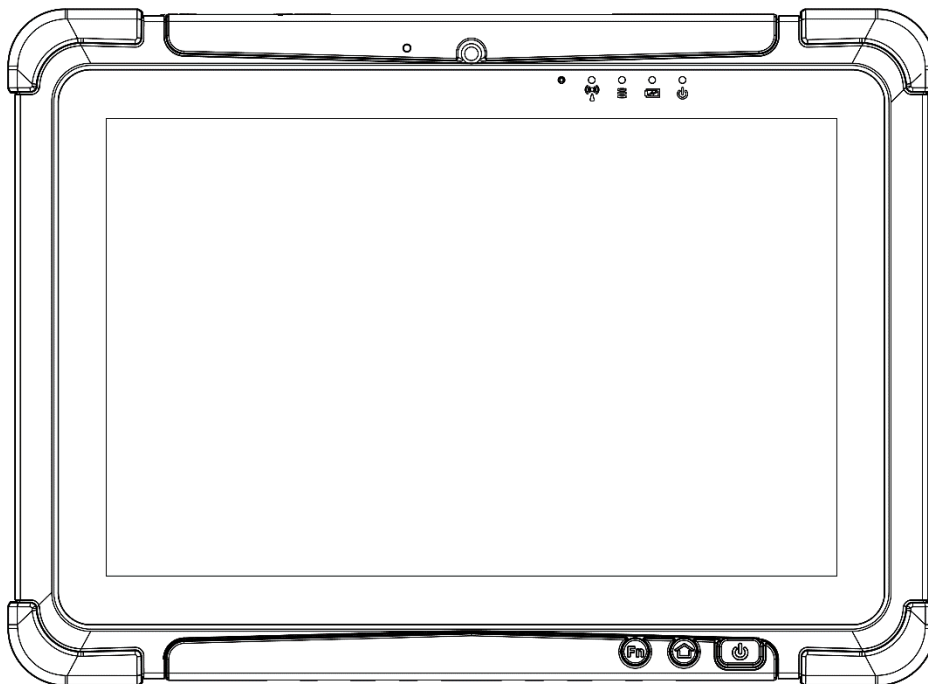
M101P-ME

Healthcare Windows Tablet

Intel® Apollo Lake Pentium N4200

Windows® 10 IoT Enterprise

Quick Start Guide

Contents

Chapter 1: Read Me First	3
1.1 Advisory Conventions	3
1.2 Intention of Use.....	3
1.3 Safety Information.....	4
1.4 Handling and Maintenance	7
1.5 Declaration of Conformity (DoC)	8
1.6 Classification	8
Chapter 2: Introduction	9
2.1 Introduction.....	9
2.2 Unpacking.....	9
2.3 Description of Parts	11
2.4 Hotkey for Specific Functions.....	13
2.5 LED Indicators	13
Chapter 3: Getting Started	14
3.1 First-Time Operation	14
3.2 Installing SIM Card (For 4G/LTE Equipped Devices)	14
3.3 Installing Battery	15
3.4 Charging Your Tablet.....	15
3.5 Starting the Tablet	16
Chapter 4: Operating the Device	17
4.1 Hottab Introduction	17
4.2 Changing Touchscreen Modes	18
4.3 Using the Camera.....	18
4.4 Using the Barcode Reader.....	19
4.5 Using the HF RFID Reader	19
4.6 Turn on/off the Light Sensor.....	20
4.7 Replacing Battery	20
4.8 Docking Connector Pin Definition.....	21
4.9 Installing Vehicle Cradle	21
4.10 Turn on/off the GPS External Antenna	22
4.11 Basic Troubleshooting	23
4.12 Screen Resolution Scaling and Setting	24
Appendix	27
Appendix A: Technical Specifications	27
Appendix C: Order Information	28
Appendix C: Meaning of symbols on the unit	28

Chapter 1: Read Me First

This chapter provides safety information, handling and maintenance precautions and other important information about using the rugged tablet.

1.1 Advisory Conventions

Four types of advisories are used throughout the user manual to provide helpful information or to alert you to the potential for hardware damage or personal injury. These are Notes, Important, Cautions, and Warnings. The following is an example of each type of advisory.

Note: A note is used to emphasize helpful information

Important: An important note indicates information that is important for you to know.

Caution: A Caution alert indicates potential damage to hardware and explains how to avoid the potential problem.

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Warning!

An Electrical Shock Warning indicates the potential harm from electrical hazards and how to avoid the potential problem.

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1.2 Intention of Use

This tablet is intended to serve as a display-integrated computing platform for integration with hospital system. This device is designed for general purpose for hospital environment. For data collection and display for reference. It shall not be used for life-supporting system.

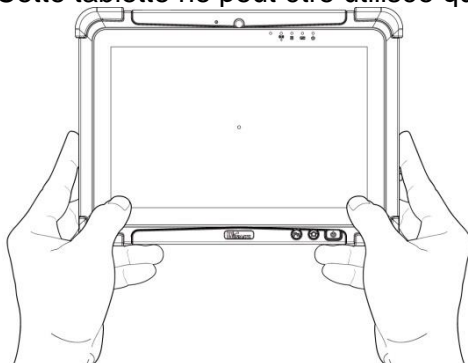
This Tablet is hand held equipment for health care professionals and can be mounted in a cradle with VESA mount for fix equipment.

Important:

This tablet can only use at landscape mode. Do not use at portrait mode.

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1.3 Safety Information

Follow and adhere to all warnings and instructions in this manual. For your safety, please read all safety and operating instructions before using the product. Keep this manual for future reference.

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Do Not Disassemble, Modify, or Insert Sharp Objects Into the Battery Pack. Electrolyte leakage, generation of heat, ignition or rupture may result.

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Do Not Short the Positive (+) and Negative (-) Terminals of the Battery. Generation of heat, ignition or rupture may result. When carrying or storing the device, do not place the battery pack together with articles that may contact electricity.

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Do Not Apply Shocks to the Battery by Dropping It or Applying Strong Pressure to It. Electrolyte leakage, generation of heat, ignition or rupture of battery may occur.

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Do Not Charge the Battery Other Than Methods Those Specified In This Manual. If the battery is not charged using one of the specified methods, electrolyte leakage, generation of heat, ignition or rupture may result.

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When the Battery Pack Has Deteriorated, Replace It with a New Battery. Continued use of a damaged battery pack may result in heat generation, ignition or battery rupture.

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Do Not Expose the Battery Pack to Excessive Heat, or Extreme Heat (Near Fire, in Direct Sunlight for example) Generation of heat, ignition or rupture may result

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Do Not Use the Battery Pack With Any Other Products. The battery pack is rechargeable and is only designed for this specific product. If it is used with a product other than the specified product, electrolyte leakage, generation of heat, ignition or rupture may result.

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Do Not Use This Product With Battery Other Than the One Specified.

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General Guideline *Directive générale*

It is recommended to reboot the device when some functions are defect or inactive. If it still can't solve the problems please contact your dealer or agent.

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Discontinue using device immediately if you notice the battery is swelling, or looks puffy or misshapen. Do not attempt to charge or use the device. Do not handle a damaged or leaking battery. Do not let leaking battery fluid come into contact with your eyes, skin or clothing.

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WARNING: To avoid risk of electric shock, this equipment must only be connected to a supply mains with protective earth.

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Warning! Do not modify this equipment without authorization of the manufacturer.

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Use a soft cloth to clean the surface.

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Please follow the local national requirement to dispose unit/accessories/waste products/residues etc.

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When you disconnect the AC adapter, disconnect from the socket-outlet First.

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The equipment should be installed near an easily accessible outlet.

For pluggable equipment, the power outlet must be installed near the equipment and must be easily accessible.

Accessory equipment connected to the analog and digital interfaces must be in compliance with the respective nationally harmonized IEC standards (i.e. IEC 60601-1 for healthcare equipment.) Furthermore all configurations shall comply with the system standard in IEC 60601-1. Anyone who connects additional equipment to the signal input part or signal output part is configuring a healthcare system, and is therefore, responsible that the system complies with the requirements of the system standard IEC 60601-1. The unit is for exclusive interconnection with IEC 60601-1 certified equipment in the patient environment and IEC 60950-1 certified equipment outside of the patient environment.

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Circuit diagrams, descriptions and component parts list will be made available only upon request when servicing is required. Please, contact the address below for related information.

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Warning! Tablet with cradle cannot be hand-held operated, and must be used with VESA mount as a fixed equipment.

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- Do not disregard the safety that a polarized or grounded connection can provide. A polarized connection consists of two plugs, one of which is larger than the other. A grounded connection has two blades and a third one that is grounded. If the main plug provided cannot be inserted into your outlet, consult an electrician to replace your obsolete outlet.
- *[FR] Ne pas ignorer la sécurité qu'une prise polarisée ou une prise de terre peut prévoir. Une prise polarisée se compose de deux fiches, dont l'une est plus large que l'autre. Une prise de terre possède deux lames et une troisième qui est la terre. Si la fiche principale fournie ne peut être insérée dans votre prise, consultez un électricien pour remplacer votre prise obsolète.*

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- Refer all servicing to qualified service personnel. A technical intervention is required when the apparatus has been damaged in any way, for example if the power cord or plug is damaged, liquid has been spilled or objects have fallen inside the unit, if the product has been exposed to rain or moisture, if it does not operate normally or if it has been dropped.
- *[FR] Confier toute réparation à du personnel qualifié. Une intervention technique est nécessaire lorsque l'appareil a été endommagé de quelque façon, par exemple si le cordon d'alimentation ou la fiche est endommagé, du liquide a été renversé ou des objets sont tombés à l'intérieur de l'unité, si le produit a été exposé à la pluie ou à l'humidité, s'il ne fonctionne pas normalement ou s'il a été jeté.*

- Grounding reliability can only be achieved when the equipment is connected to an equivalent receptacle marked "Hospital Only" and "Hospital Grade". Use a power cord that matches the voltage of the power outlet, which has been approved and complies with the safety standard of your particular country.
[FR] La fiabilité de la mise à la terre ne peut être atteinte que lorsque l'équipement est connecté à un réceptacle équivalent portant la mention «Hospital Only» et «Hospital Grade». Utilisez un cordon d'alimentation correspondant à la tension de la prise de courant approuvée et conforme aux normes de sécurité de votre pays.

1.4 Handling and Maintenance

General Guideline *Guide Général*

Keep the product at least 13 cm (5 inches) away from electrical appliances that can generate a strong magnetic field such as TV, refrigerator, motor, or a large audio speaker.

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Do not place heavy objects on top of the tablet computer as this may damage the display.

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The screen surface can be easily scratched. Use with care. Do not use paper towels to clean the display but use the included cleaning cloth. Avoid touching it with a pen or pencil.

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To maximize the life of the backlight in the display, allow backlight to automatically turn off as a result of power management. Avoid using a screen saver or other software that prevents the power management from working.

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Cleaning Guideline *Guide de nettoyage*

Use a soft cloth moistened with water or a non-alkaline detergent to wipe the exterior of the product.

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Gently wipe the display with a soft, lint-free cloth. Do not use alcohol or detergent on the product.

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Use your finger or the stylus on the display. Using sharp or metallic object may cause scratches and damage the display, thereby causing errors.

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Use a soft cloth to remove dirt on the display. The touchscreen surface has a special protective coating that prevents dirt from sticking to it. Not using a soft cloth may cause damage to the special protective coating on the touchscreen surface. To clean off hard to remove dirt, blow some steam on the particular area and gently wipe away using a soft cloth.

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Do not use excessive force on the display. Avoid placing objects on top of the display as this may cause the glass to break.

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When there is a noticeable discrepancy in the operation of the touchscreen function (your touch does not correlate to the proper location on the screen), recalibrate the touchscreen display.

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1.5 Declaration of Conformity (DoC)

FCC Statement

This device complies with part 15 FCC rules.

Operation is subject to the following two conditions:

- This device may not cause harmful interference.
- This device must accept any interference received including interference that may cause undesired operation

This equipment has been tested and found to comply with the limits for a class "B" digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

CE Notice (European Union)

Healthcare electrical equipment Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances

- EN 60601-1-2:2015
- EN 5511:2019+A1:2010
- EN 61000-4-2: 2009
- EN 61000-3-2:2014
- EN 61000-4-3: 2006+A1: 2008+A2: 2010
- EN 61000-3-3:2013
- EN 61000-4-4: 2012
- EN 61000-4-5: 2006
- EN 61000-4-6: 2014+AC:2015
- EN 61000-4-8: 2012
- EN 61000-4-11: 2004

Healthcare electrical equipment Part 1: General requirements for basic safety and essential performance

- EN 60601-1:2016/A1:2013/A12:2014

This equipment is in conformity with the requirement of the following EU legislations and harmonized standards. Product also complies with the Council directions.

1.6 Classification

Supply Class I adapter
 No applied part
 IP65 (Only for panel)
 Continuous Operation
 Not AP or APG category

Chapter 2: Introduction

This chapter describes basic information of the M101P-ME Healthcare Tablet.

2.1 Introduction

Thank you for purchasing 10.1-inch Healthcare Windows Tablet Computer M101P-ME. The M101P-ME is Windows-based tablet computer designed for healthcare applications with a housing enhanced with antimicrobial properties, weighting at just 1200 grams (2.7 pounds). This mobile and portable device is flexible enough to function as a standalone device and competent enough to be used for all sorts of data collection, patient care, information management and wireless communication in healthcare applications. With rugged features, a crystal clear 1920 x 1200 resolution display, projected capacitive touch screen, barcode scanner and superior processing power coming from Intel's Apollo Lake processor, this tablet can be used in hospital halls, waiting rooms or nurse stations for communicating and documenting patient information, allowing paramedics and nurses to instantly access and keep patient records.

Highlights

- 10.1" 1920 x 1200 IPS LED Panel with direct optical bonding
- Intel® Apollo Lake Pentium N4200 processor
- Housing enhanced with antimicrobial properties
- Long battery life
- USB 3.0 (Type C) connector
- Optional 1D/2D barcode scanner to scan medications or wrist bands
- Lightweight, rugged, IP65 waterproof and dustproof
- IEC 60601-1, 60601-1-2 Certified for healthcare

2.2 Unpacking

Check and identify the standard supplied accessories.

- **Healthcare Tablet Computer**

- **Power Adapter**

Manufacturer : EDAC POWER ELECTRONICS CO., LTD.
Model name : EM10683V & EM10681V

- **Standard Battery**

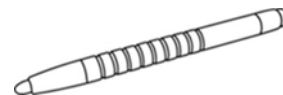
Manufacturer: Winmate Inc.
Model name : BS101

- **Power Cord and Plugs**

Manufacturer : PHINO ELECTRIC (HUIZHOU)LTD.
Model name : PHP-350 TO PHS-301

- **Capacitive Touch Stylus**

Manufacturer: HuntWave Industrial Co., Ltd.
Model name : 3.5mm Plastic pen tip and 5.5mm Rubber stylus

- **Driver CD**

Optional Accessories

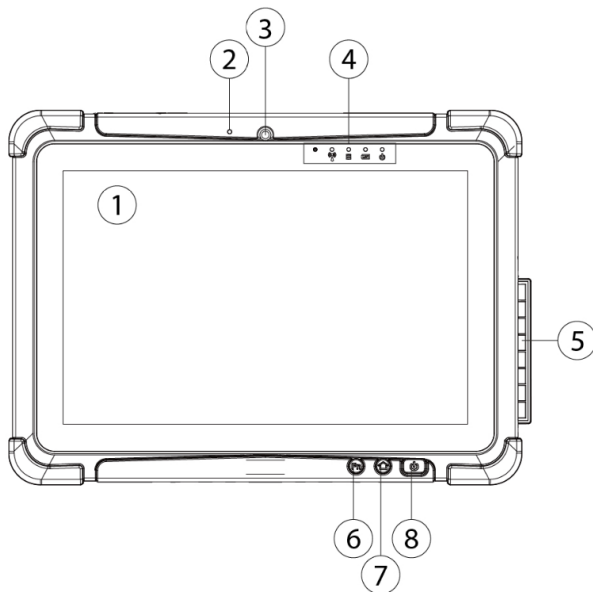
Your package may include optional accessories based on your order.

- Healthcare Grade Tablet Dock (Designed for Healthcare Applications)
- Desk Dock
- Vehicle Dock
- Battery Charger
- Smart Card Reader
- Handstrap
- UHF RFID Reader
- VESA Mount Kit
- Shoulder Strap
- Carry Bag
- Micro HDMI Cable
- Vehicle Charger

2.3 Description of Parts

Dimensions: 271.8 x 197.2 x 19 mm


Front:

Left:

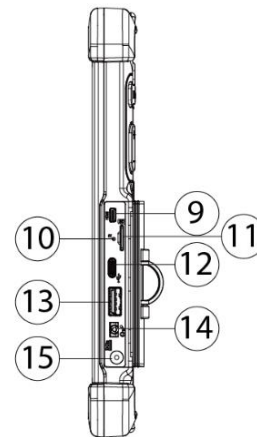
Right:


No	Item	Description
1	10.1" Touch Screen	Display and act as one of the inputs for the Tablet Computer
2	Digital Mic with noise cancellation	Use during video recordings and communications.
3	Front Camera	2 Megapixel Webcam
4	Light Sensor and LED Indicators	Show the current status of the Tablet Computer. RF communications, storage, battery status, power status.
5	I/O Cover	Protect I/O ports
6	Fn1 Button	Programmable function key configured by Hottab Utility
7	Menu Button	Hottab Quick Access
8	Power Button	Power on/off tablet

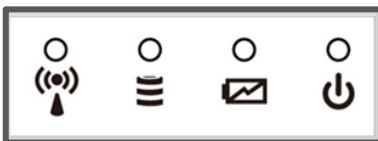Front LED Indicator Details:


No	Item	Description
9	Micro HDMI	HDMI output to external display
10	Clear CMOS	Hardware reset button.
11	MicroSD Card Reader	Memory card reader port for MicroSD Card
12	USB 3.0 Type C	High speed USB port connects to USB devices, such as USB flash drive, mouse, keyboard, printer, etc. Note: The USB Type-C connector follows USB 3.0 standard, which does not support Alternate Mode (ALT) or USB Power Delivery (USB-PD).
13	USB 3.0 Type A	USB 3.0 port connects to USB devices, such as USB flash drive, mouse, keyboard, printer, etc.
14	Mic In/Line Out	Headset or headphone can be connected (Mic is only supported with 3.5mm 4-pin headset)
15	Power Jack	DC power input jack.

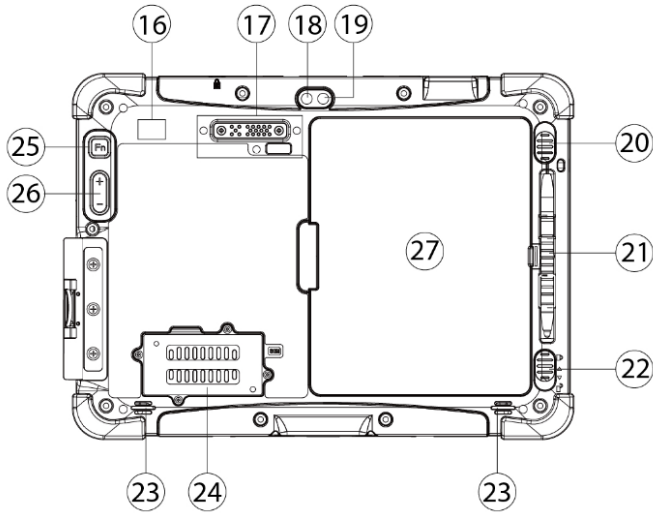
Rear:

Top:

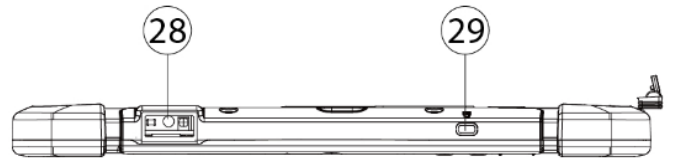
Bottom:

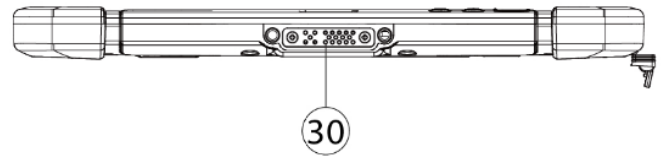No	Item	Description
16	Optional RFID Reader	RFID Reader for reading RFID tags.
17	Optional Extension Port	Provides connectivity to custom modules for special applications (USB2.0 x 1, RS232 x 1).
18	Camera Flash	LED camera flash to provide flash lighting when use with camera.
19	Rear Camera	8MP Autofocus Camera. Captures photos and videos.
20	Battery Release Slider	Spring-loaded slider that latches the battery in.
21	Touch Pen Slot	Pen holder slot to store touch pen
22	Battery Lock Latch	Latch to lock the battery in place once battery is inserted. Not spring-loaded.
23	Speakers	Emit sounds.
24	4G / LTE Module / SIM card slot	Service window to insert Micro SIM card. Also provide access to 4G / LTE Module.
25	Fn2 Button	Programmable function key configured by Hottab.
26	Brightness or Volume Buttons	Brightness/ Volume Control. (+) Up, (-) Down.
27	Battery	7.4V, typ. 5140 mAh Li-Polymer Battery (2S1P) 7.4V, typ. 10280 mAh Li-Polymer Battery (2S2P, optional)

No	Item	Description
28	Optional Barcode	Barcode Scanner for reading of 1D/2D barcode.
29	Kensington Lock Port	Provide lock access for Kensington Lock.
30	Docking Connector	Port to docking solutions such as vehicle docking to provide power and external antenna.

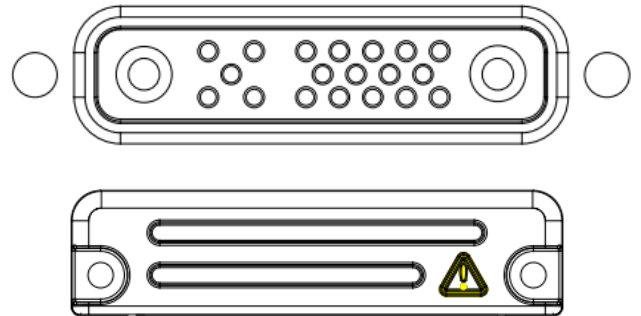Below is the expansion port and the expansion port cover.

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2.4 Hotkey for Specific Functions

For the physical buttons on the tablet computer, in addition to its dedicated functions detailed in the previous section, it can also be used to emulate commonly used keyboard key strokes. Under different scenario, a single physical button or a combination of physical buttons provide different keyboard key strokes, which are specified below:

In POST (before Windows boots up):

Buttons	Keystroke
Fn1	F6 to enter Recovery Menu
Fn2	Esc to enter BIOS setup screen
Up (+)	Up
Down (-)	Down
Menu + Fn1	Enter
Menu + Up (+)	Right
Menu + Down (-)	Left

In Windows:

Buttons	Keystroke
Menu	Hottab Quick Access
Fn1	Default: Web Browser
Fn2	Default: Camera
UP (+)	Up.
Down (-)	Down.
Power	Power ON/ OFF
Up (+) +	Ctrl + Alt + Del
Down (-)	

2.5 LED Indicators

Item		Description
Power Indicator		Blue – Tablet is ON No light – Tablet is OFF
Battery Status		Orange –AC adapter is plugged in and charging Green –AC adapter is plugged in and battery is full charged Slow Blinking Orange - Low battery (<10%) Fast Blinking Orange – Battery is being hot swapped. No light – Battery is in use and more than 10%
Disk Activity Indicator		Green - Disk Activity No light - Disk Idling
RF Communication		Green - WiFi/BT/WWAN/GPS ON No light - WiFi/BT/WWAN/GPS are all OFF

Chapter 3: Getting Started

This chapter tells you step by step how to get the Rugged Tablet up and running.

3.1 First-Time Operation

To get started:

1. Install SIM card.
2. Install storage card.
3. Install the battery.
4. Charge your tablet.
5. Start the tablet.

3.2 Installing SIM Card (For 4G/LTE Equipped Devices)

The Tablet Computer supports 4G/LTE data connection and fits a Micro-SIM card.

To install the SIM card:

Tools Needed:

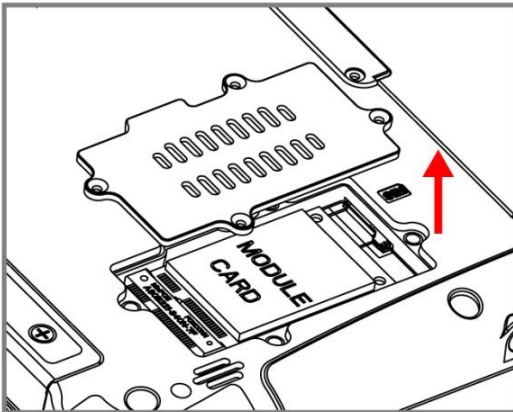
Service Windows

Screw: M2x5, 4 pc
Screwdriver: Philips Type , Size #0 , for
M2 screws, torque: 2.0 ± 0.5 kgf.cm


4G/ LTE Module

Screw: M2x3.5, 1 pc
Screwdriver: Philips Type , Size #0 , for
M2 screws, torque: 2.0 ± 0.5 kgf.cm


- 1** Remove four screws and remove the service window.

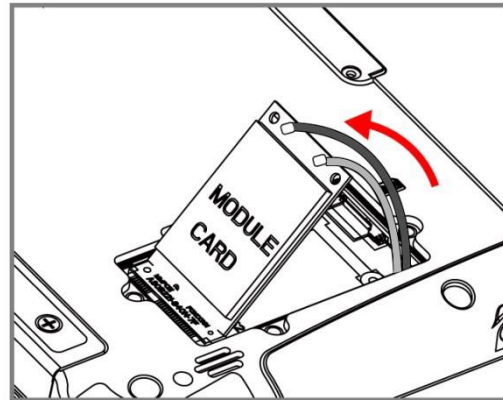
- 2** Remove the one screw that secures the 4G/ LTE module. Carefully lift the module up. **Do not to damage the connected cables!**

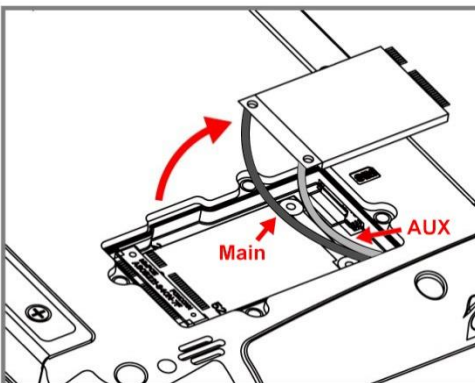
- 3** Carefully flip and move the 4G/ LTE module and to the right. **Do not disconnect cables!**

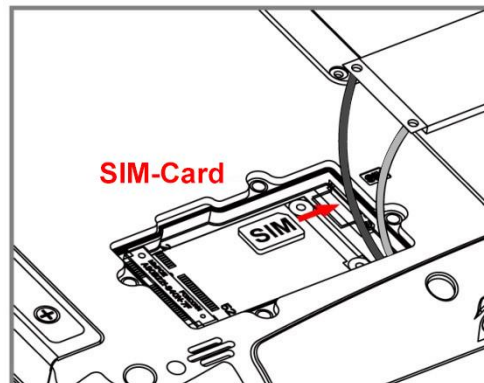
- 4** Insert SIM-Card all the way in. **Chip/ contact facing down.**

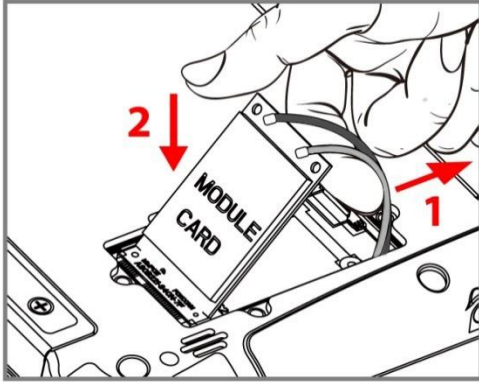- 5 Use your hand to carefully pull the cable outside the module card. Re-insert the 4G/LTE module and fasten one screw. Accurately route the cables. **Do not bend cables! Do not pinch/squeeze the cable!**

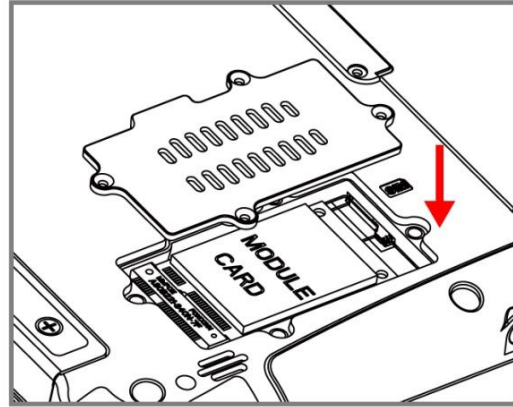- 6 Re-insert the service window and fasten with the four screws.



Caution

When re-inserting 4G/LTE module, please pay attention to cable routing. Make sure the cable module connects properly and does not get squeezed.

3.3 Installing Battery

To install the standard battery pack:

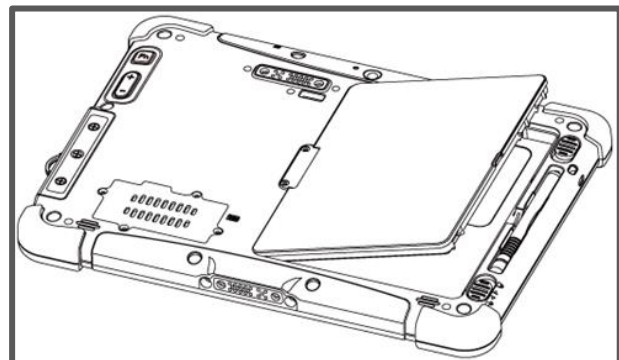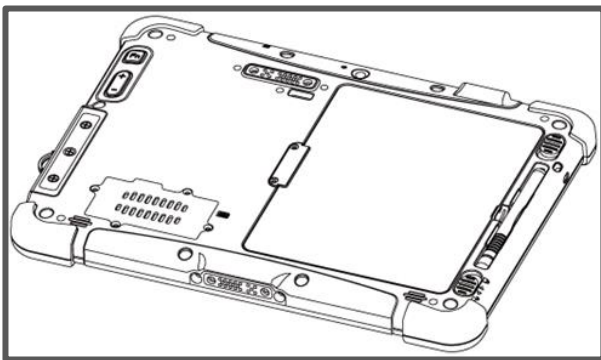1. Disconnect any cords connected to the tablet computer.
2. Turn the tablet over, back facing up.
3. Locate the Battery Lock Latch and push it to the “Unlocked” position.
4. Locate the Battery Release Slider that is spring-loaded. Pull back and hold.
5. Insert the battery with the connector-pins side first, and then lower the battery into the battery compartment. Continue to hold the Battery Release Slider until the battery is fully inserted.
6. Make sure the battery is completely lowered into the compartment before letting go of the Battery Release Slider. Battery will be flat against the housing once fully inserted.

To remove the battery pack:

1. Shut down the tablet (or put into sleep mode). Disconnect any cords connected to the computer.
2. Turn the tablet computer over, back facing up.
3. Locate the Battery Lock Latch and push it to the “Unlocked” position.
4. Locate the Battery Release Slider that is spring-loaded. Pull back and hold.
5. Lift the battery out using the opening on the side of battery compartment.

3.4 Charging Your Tablet

The M101P-ME Healthcare Tablet Computer operates either on external AC power or internal battery power. It is recommended to use AC power when you start up the Tablet for the first time.

Caution: Use only the AC adapter included with your Tablet Computer. Using other AC adapters may damage the Tablet Computer and the Battery.

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Caution: For users start up the Tablet Computer for the first time with internal battery power, please make sure the battery is fully charged before using.

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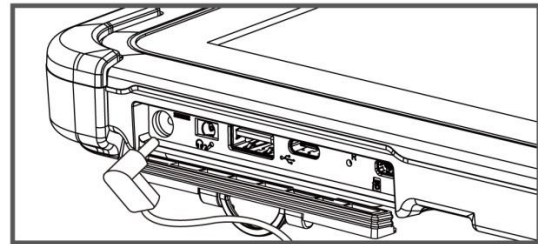
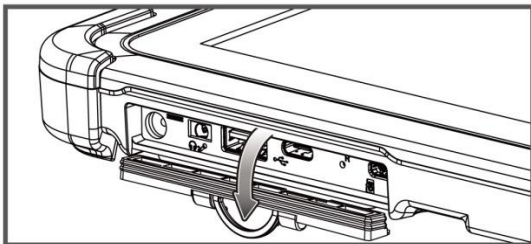
To connect a tablet to AC power:

1. Open I/O protective cover by pulling out the latch.
2. Plug the DC cord of the AC adapter into the power jack of the tablet computer.
3. Plug the AC power cord into the AC adapter.
4. Plug the AC power cord into the electrical outlet.

Item	Battery Operation Time	Battery Charging Time
Standard Battery (BS101)	8 Hours	2.5 Hours
High Capacity Battery (BL101)	16 Hours	4.5 Hours

3.5 Starting the Tablet

Important: When powering on the device for the first time, please make sure there's no interruption during the Windows setting up process, for example, accidentally powering off. If it happened, please access the recovery menu by pressing **Fn1** button on the front bezel to initiate recovery process.

Press and hold the power button for **3 seconds** until the blue LED power indicator light up.

Note: Do not press the power button repeatedly.

Do not perform the following operations until the Disk Activity Indicator off

- Connecting or disconnecting the AC adapter
- Pressing the power button
- Touching the tablet buttons, screen, or external keyboard/mouse

To power off the Tablet Computer:

1. Make sure you have closed any programs that are open on the Desktop. Start Screen apps don't need to be closed.
2. Tap on the **Settings** charm.
3. Tap on **Power**, then tap on **Shut Down**.
4. Wait for the Tablet Computer to power off.

Note: The computer will forcibly be powered off if the power button is pressed and held for **6 seconds or longer**.

Chapter 4: Operating the Device

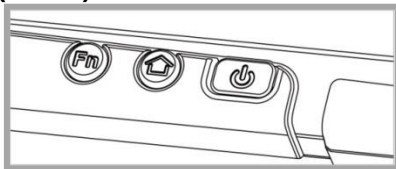
This chapter describes how to operate the M101P-ME Tablet Computer, including camera, Hottab, barcode, RFID, WWAN, WLAN, GPS features.

4.1 Hottab Introduction

Hottab provides quick access to the tablet's functions and features, such as setting up shortcuts, using camera, powering on/off built-in devices, programming function key, and changing power management.

As Windows starts up, Hottab starts and runs in the background. To display the Hottab control utility, press the  (Menu) Button located on the front panel, OR launching the Hottab Utility application.

 (Menu) Button on the Front Panel:

Hottab Utility Icon:

The main user interface is located on the right side of the screen. There are 4 main menu options: Shortcuts, Camera, Device ON/OFF, and Settings. Operations and functions on each menu options is explained in detail in the coming section.

Icon	Description
 Shortcuts	Allows users to configure short cut to quickly access programs with a few tabs.
 Camera	Allows users take a picture or record a video. User can also adjust setting related to camera in this menu option.
 Device ON/OFF	Enables/disables tablet computer's hardware functions. The configurable functions are Wi-Fi, WWAN (4G/LTE), GPS, Bluetooth, Camera, LED indicators, Barcode Reader, RFID Reader, and GPS Antenna routing options.
 Settings	Provides users information such as PC Information, Battery status, and also device settings such as power management setting, and function key mappings.

4.2 Changing Touchscreen Modes

The M101P-ME Healthcare Tablet Computer supports three built-in operating modes for different usage scenarios:

Mode	Usage Scenario
 <p>Hand/Rain (Default)</p>	<p>In <i>Hand/Rain</i> mode the touchscreen rejects false inputs from liquid drops. When the liquid is removed, the touchscreen accepts touch inputs from bare fingertips.</p> <p><i>Supports input by:</i></p> <ul style="list-style-type: none"> • <i>Rubber tip passive stylus</i> • <i>Bare fingertips</i>
 <p>Stylus</p>	<p>In <i>Stylus</i> mode, the touchscreen supports inputs from the fine-tip passive capacitive stylus (shipped with tablet).</p> <p><i>Support input by:</i></p> <ul style="list-style-type: none"> • <i>Fine-tip passive stylus with palm rejection</i> • <i>Rubber tip passive stylus</i> • <i>Bare fingertips</i>
 <p>Glove</p>	<p>In <i>Glove</i> mode, the touchscreen supports inputs while the user is operating the touchscreen with gloves on.</p> <p><i>Supports input by:</i></p> <ul style="list-style-type: none"> • <i>Glove</i> • <i>Rubber tip passive stylus</i> • <i>Bare fingertips</i>

4.3 Using the Camera

To take photos or videos:

1. Open the Hottab application. The on-screen Hottab Camera Indicator lights orange.
2. Point the camera at an object.
3. Touch the on-screen shutter button  to take a photo or video record  button to record a video.

Item		Description
Review		Review
Settings		<p>Opens Camera settings page.</p> <p>Available Settings:</p> <ul style="list-style-type: none"> • Picture Save: modify/inspect saved location for pictures. • Video Save Folder: modify/inspect saved location for video recordings. • Picture Quality: Picture compression quality.
Camera Switch		Switch between front camera or rear camera.
Flash Control		Enables/ Disables LED Flash when taking picture.
Video Record		Tap to Start, and press again to stop recording.
Shutter Camera		Camera Shutter Button. Tap to take picture.
Exit		Exits the Camera and hide Hottab Utility.

4.4 Using the Barcode Reader

The 2D/1D Barcode Reader is optional for M101P-ME Tablet PC and is not re-workable by the end user. To use this feature, please make sure the tablet PC is built-in barcode reader before use.

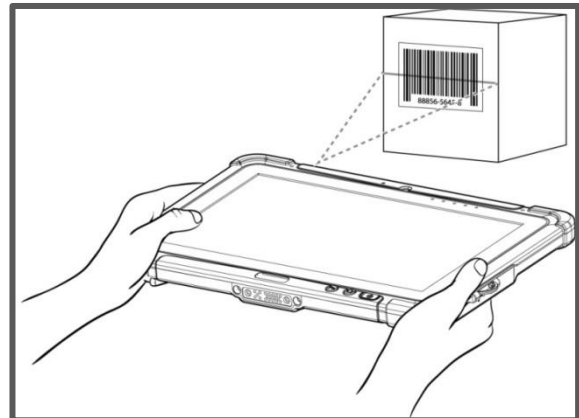
Caution: Use of controls or adjustments or performance of procedures other than those specified herein may result in hazardous radiation exposure.

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1	Wavelength	630-680nm
2	Beam divergence	2mrad (Horizontal) x 1.3mrad (Vertical) expressed at 1/e ² (13.5%) intensity level.
3	Pulse duration and repetition rate (or description of irregular pulse pattern)	1.68mSec. Irregular pulse.
4	Maximum power or energy output	1mW

To set up a barcode scanner:

1. Tear off protective film from the barcode.
2. Press  Menu to open Hottab utility.
3. Choose Device ON/OFF  and click on Barcode .
4. Tap barcode power ON.
5. Press **Fn** button located on the rear.
6. Scan a barcode to check your reader is working properly. Result will be displayed in Hottab.

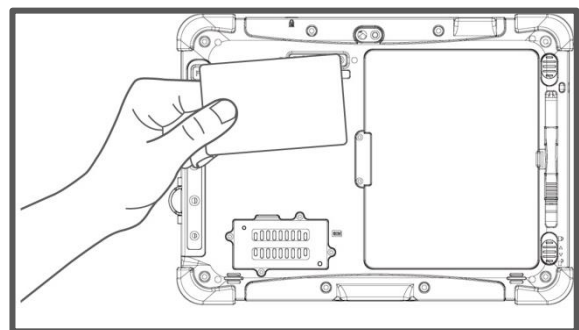
4.5 Using the HF RFID Reader

The HF RFID Reader is optional for M101P-ME Tablet PC and is not re-workable by the end user. To use this feature, please make sure the tablet PC is built-in HF RFID reader before use, or contact our sales contact window for more details.

To enable RFID Reader:

1. Press  Menu to open Hottab utility.
2. Choose Device ON/OFF  and click on .
3. Tap on the ON button. The button turns green, and the RFID icon in the Hottab menu is orange.
4. Bring the RFID card toward the antenna. Result will be displayed in Hottab

4.6 Turn on/off the Light Sensor

The Light Sensor can be turned on/off from Hottab menu. To enable/disable the Light Sensor, perform the following procedure:

1. Open the Hottab utility by press the  (Menu) Button located on the front panel, OR launching the Hottab Utility application.
2. Tap on the **Device ON/OFF** button.
3. Check the status color of the **Light Sensor** icon:

Item	Status Color	Description
 Light Sensor	Orange	Light Sensor is enabled.
	White	Light Sensor is disabled.

4.7 Replacing Battery

The M101P-ME Tablet supports hot-swap, meaning the tablet doesn't need to be powered off to replace the main battery. To support hotswap, there is a backup battery inside the tablet that is charged together with the main battery. This battery cannot be removed, and requires adequate power for main battery hot swapping. When the backup battery is fully charged, the tablet can sustain up to 5 minutes of runtime for battery hot-swapping.

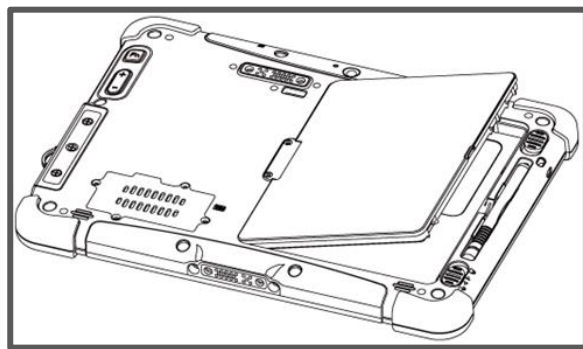
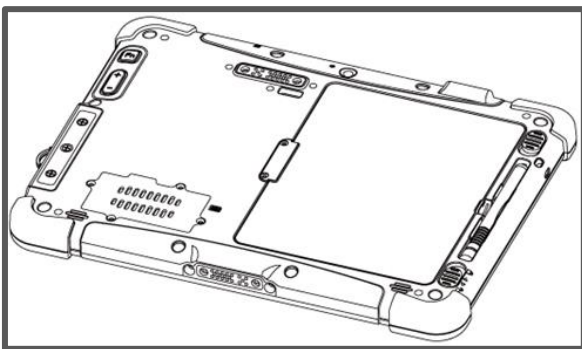
Perform the following to hot swap the batteries:

1. Power on the tablet, open Hottab to check the power level of the Backup battery.
2. If the Backup battery level is "Low", please charge the tablet with AC Power until the Backup battery level is "Medium" or "High", and do not remove the main battery.
3. If the Backup battery level is at "Medium" or "High", you can remove the main battery.
4. Insert a new fully charged battery into the battery compartment.
5. Pen Hottab to check Main battery level. If the battery level is displayed incorrect, check if the new removable battery is inserted properly.
6. If the battery level displayed correct, continue using the tablet.


Caution: Charge the built-in back-up battery every two months to keep battery health and prevent possible issues from battery over-discharge.

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4.8 Docking Connector Pin Definition

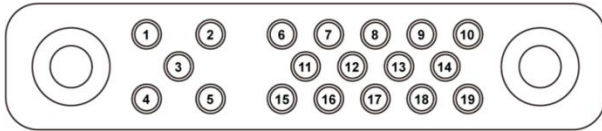The M101P-ME Healthcare Tablet Computer supports one 19-pin docking connector at the bottom side. The pin definition is listed as the following table.

19 Pin Docking Connector			
Pin No.	Description	Pin No.	Description
1	RF_GND	11	NC
2	DOCK_DET#	12	NC
3	GPS ANT	13	GND
4	RF_GND	14	Power
5	RF_GND	15	USB+
6	NC	16	USB-
7	NC	17	GND
8	DETECT+	18	GND
9	Power	19	DETECT-
10	Power		

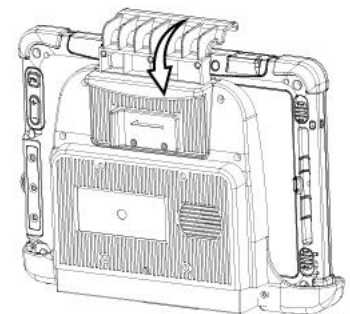
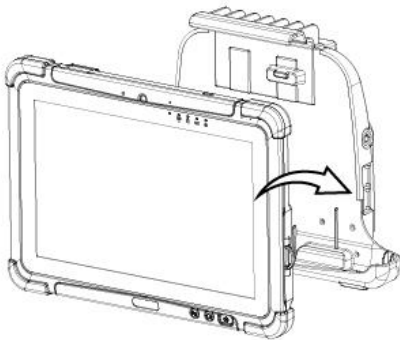
Caution: Do not open when the expansion port is not used

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4.9 Installing Vehicle Cradle

Use a vehicle cradle to mount a tablet on the vehicle. Follow the instructions below to install M101P-ME healthcare tablet on the vehicle cradle.

- Align the cradle alignment fingers on the bottom of the cradle mount to bottom of the tablet, and then push down the tablet in against the cradle mount.
- Make sure the tablet's back is aligned upright against the wide range power input mount. Reposition if the tablet still moves around.
- Push down the hook until a click sound is heard to secure the tablet into the Cradle mount.




How to remove the tablet from the cradle

① Reach for the latch on the back of the tablet and pull it sideways.

② The top hook should be released after the latch is pulled.

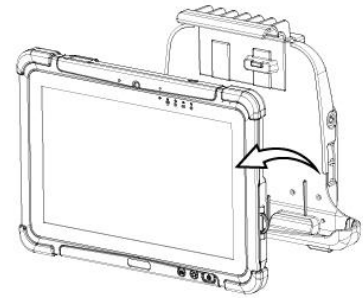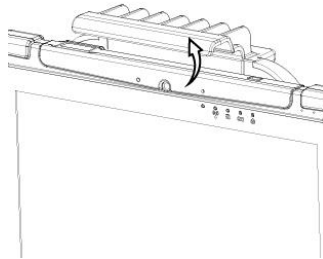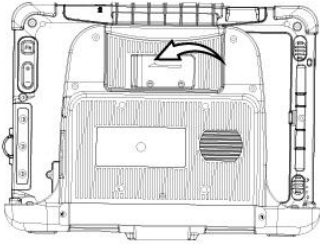③ Tilt the tablet slightly forward first, and then pull the tablet straight up to remove it.

The Cradle’s input/output expansion functions will only work if power is connected. Provide 19V input power to the terminal block directly.

External Display Modes (If optional VGA port is equipped)

Display Modes	Description
Mirror Mode (Default)	The main display resolution might be affected by extended monitor. Follow the scenarios in installation guide to reset the display.
Extend Mode	It is recommended to operate the tablet with keyboard / mouse under this mode. If touch function is desired, please use the mirror mode.

Note: Display settings are saved automatically for use next time.

4.10 Turn on/off the GPS External Antenna

The vehicle dock of M101P-ME Tablet PC supports external GPS antenna through standard SMA connector. To work with external GPS antenna, please make sure the tablet PC is well installed to vehicle dock, and the vehicle dock has connected with external GPS antenna.

When working with vehicle dock connecting to external antenna, user can switch GPS antenna from the internal one to the external one from Hottab menu.

To enable/disable the GPS Ext. Antenna:

1. Open the Hottab utility by press the  (Menu) Button located on the front panel, OR launching the Hottab Utility application.
2. Tap on the **Device ON/OFF** button.
3. Check the status color of the **GPS Ext. Antenna** icon:

Item	Status Color	Description
 GPS Ext. Antenna	Orange	GPS Ext. Antenna is enabled, and using external antenna if available
	White	GPS Ext. Antenna is disabled, and using built-in antenna

4. If the GPS Ext. Antenna is disabled (white), tap on the GPS Ext. Antenna icon to enable it (orange).

4.11 Basic Troubleshooting

Problem	Solution
The removable battery is not charging.	<ul style="list-style-type: none"> • Make sure the AC adapter is properly connected, and that the removable battery is properly inserted.
Nothing appears on the screen.	<ul style="list-style-type: none"> • The tablet computer may be in sleep mode. Press the power button to wake up the display. • The brightness level may be too low. Increase brightness.
The screen suddenly turns blank and the power indicator goes off.	<ul style="list-style-type: none"> • Make sure that LED indicator is not powered off in Hottab. • Make sure the AC adapter is properly connected. • If running on battery power, the battery may be depleted, charge the battery.
Wireless LAN signal quality is poor.	<ul style="list-style-type: none"> • The tablet computer may be out of range. Move the tablet computer closer to the access point or another wireless LAN device associated with it. • Check if your surroundings have interferences, such as microwave ovens, cordless phones. Move the tablet computer away from objects causing interference.
I cannot connect to another wireless LAN device.	<ul style="list-style-type: none"> • Make sure that wireless LAN feature is powered on. • Make sure that the SSID setting is the same for every wireless LAN device in the network. • Restart the tablet computer • Make sure the IP address or subnet mask setting is correct.
I cannot connect to another device with Bluetooth.	<ul style="list-style-type: none"> • Make sure that both devices have Bluetooth powered on (For the Tablet Computer, check Hottab to see if Bluetooth is powered on). • Make sure that the distance between the two devices is within 10 meters and there are no walls or large obstructions between the devices. • Make sure the both devices are powered on and configured to be discoverable. • Make sure that both devices are compatible.
The GPS application cannot find my location.	<ul style="list-style-type: none"> • Make sure that GPS is powered on in Hottab. • Make sure that GPS receiver and the location sensor are enabled in Control Panel. • The tablet computer may be out of coverage area. Move your tablet computer outdoors or closer to a window. • Check if your surroundings have interferences, such as microwave ovens, cordless phones. Move the tablet computer away from objects causing interference.
No sound is produced.	<ul style="list-style-type: none"> • Adjust volume of the tablet computer by pressing on the volume rocker. • If using an external audio device, make sure that the cable is properly connected.
I cannot project the tablet computer display to another monitor.	<ul style="list-style-type: none"> • Make sure the VGA driver is installed. • Make sure the output setting selected is correct.

Problem	Solution
Barcode scanner does not emit laser beam and does not seem to work.	<ul style="list-style-type: none"> • Make sure the Barcode scanner device is enabled in Hottab. • Make sure under Windows, the Barcode Reader is set to COM 15 port, so that Hottab can trigger the Barcode Reader hardware.
Barcode Scanner emits laser beam but does not read barcode presented to it.	<ul style="list-style-type: none"> • Adjust the distance between barcode scanner of the device and the barcode being scanned. • Make sure the barcode code type is enabled in the tablet. To enable/disable a specific barcode code type, go through the “Barcode Scanner Configuration” section.
I am having GPS accuracy issue.	<ul style="list-style-type: none"> • When utilizing the GPS function, we recommend disabling the Barcode Reader in Hottab to reduce interference.
RFID scanner does not seem to work.	<ul style="list-style-type: none"> • Make sure under Windows, the RFID hardware is set to COM 3 port.
Battery leaking or swollen.	<ul style="list-style-type: none"> • The lithium battery used in this tablet may swell after being charged/ discharged many times, but there is no problem regarding safety. • Continued use in such a state may result in the battery getting jammed, so please stop using the battery, and purchase a new one.

4.12 Screen Resolution Scaling and Setting

The M101P-ME has a 1920 x 1200 high resolution display is perfect for viewing image, video, or other applications, however, Windows user interface (icons, buttons, and other user interface elements) may be too small to read and operate at the default Windows settings. By changing the display scaling or the display resolution setting of the device, you can adjust the size of the text and other items on the desktop for optimal viewing. Display scaling is the recommended method because it has the effect of adjusting user interface element without having to move away from the optimal resolution setting at 1920 x 1200. This section will explain how you can adjust the built-in windows display scaling setting and resolution setting.

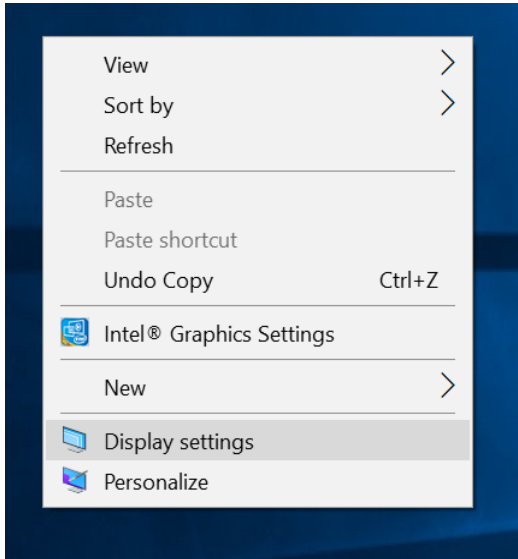
Note:

- The M101P-ME Healthcare Tablet computer is shipped with display scaling set at **Larger - 150%**. We recommend the users set the scaling to this setting for optimal text and image viewing.
- Before proceeding to change the scaling option, make sure any opened document is saved and close, and all running applications are closed before proceeding to change the display scaling, because this procedure requires users to log out of their account to apply display scaling change.


Changing Display Scaling (Recommended)

Perform the following to change display scaling setting:

1. Tap and hold anywhere on the desktop for 3 seconds and release to bring up the context menu, and select “Display settings” option as shown below:

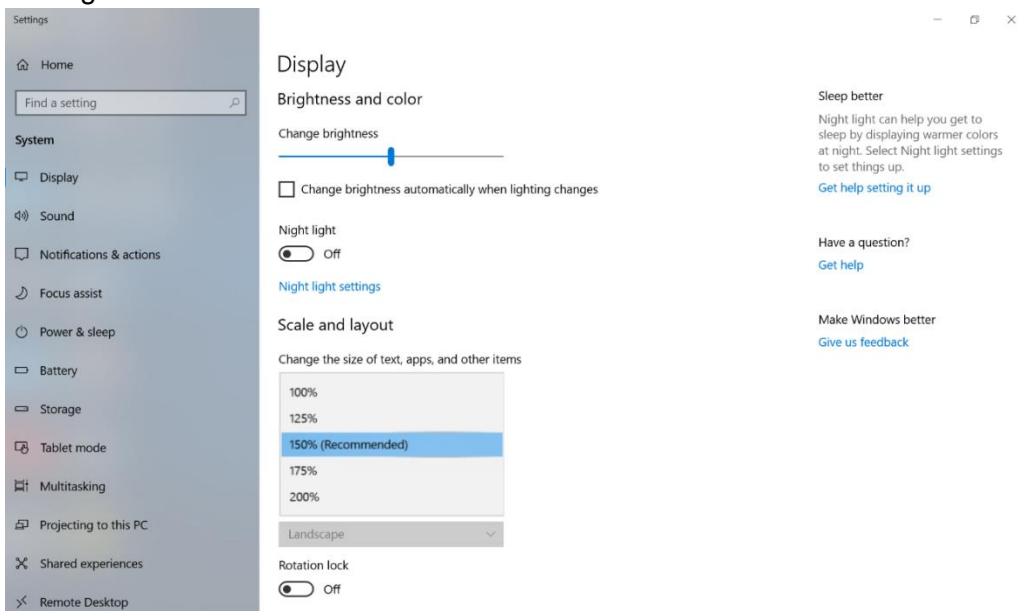
2. A “Display” Window will show up. Click on the dropdown option under the “**Scale and layout**” section and change the scaling option to “**150% (Recommended)**” or any other desired display scaling.


Changing Screen Resolution

Alternatively, instead of changing the display scaling setting, user may choose to change the overall resolution setting to make screen elements bigger.

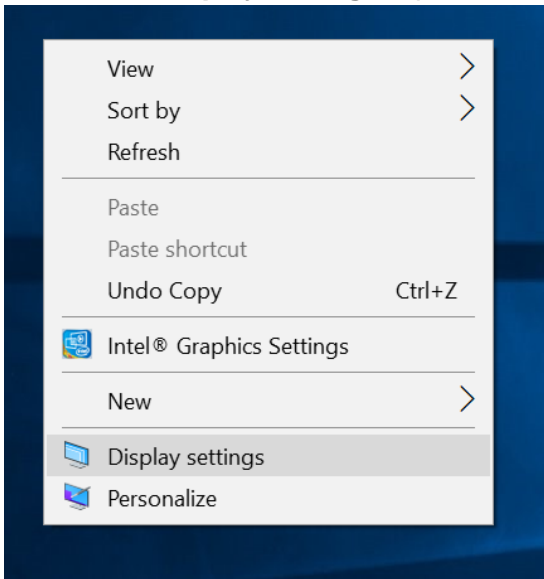1. Tap and hold anywhere on the desktop for 3 seconds and release to bring up the context menu, and select “**Display settings**” option as shown below:

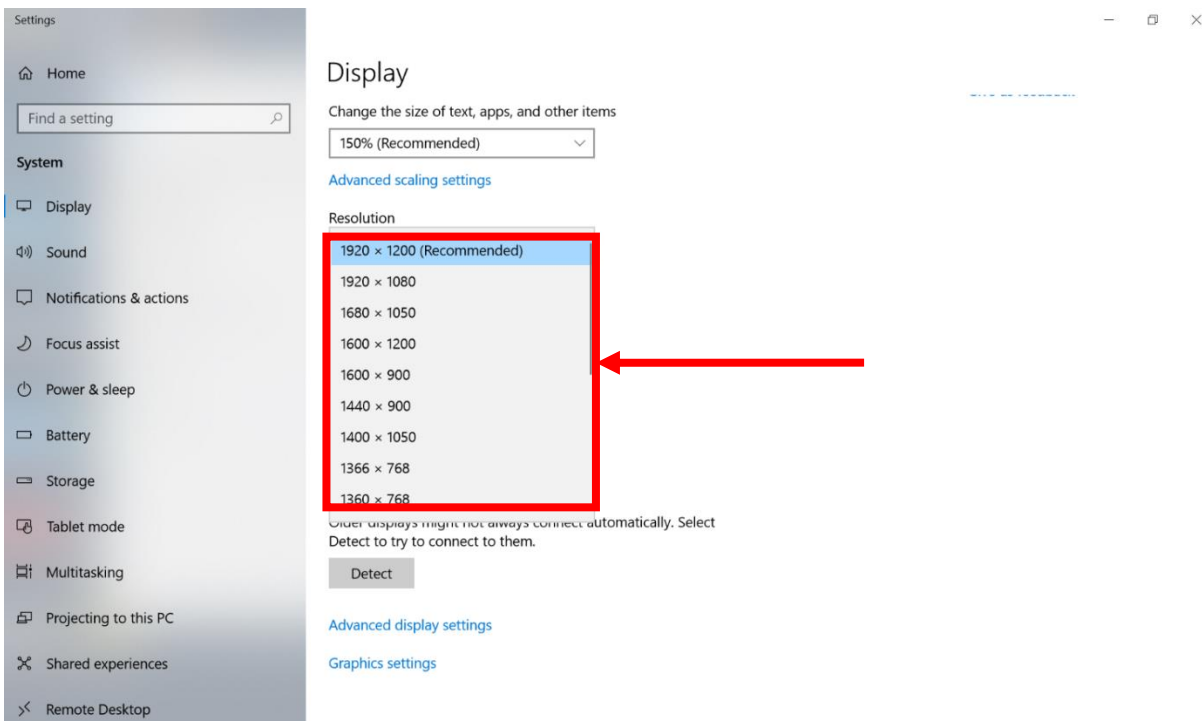2. Click on the dropdown option next to “**Resolution**” and select the desired resolution by tapping on the scale:

3. Hit “**Apply**”. A prompt “**Display Settings**” prompt will show up to ask the user if the applied resolution is usable.
4. If the new resolution is the desired display setting hit “**Keep changes**” and continue to next step. If the new resolution is not desirable, hit “**Revert**” and select a different resolution to change to, and start again from step 2.
5. The user may now use the device with the new resolution setting.

Appendix

Appendix A: Technical Specifications

		Model Name
		M101P-ME
Display Specification	Size	10.1-inch IPS display with LED backlight
	Resolution	1920 x 1200 WUXGA
	Brightness	600 nit
	Contrast Ratio	800:1
	Viewing Angle	85/85/85/85
	Technology	<ul style="list-style-type: none"> • Anti-glare technology • Ambient light sensor, e-compass, gyro, acceleration sensors • Automatic screen rotation • Intel HD Graphics (built-in CPU) video controller
System Specification	CPU	Intel Pentium N4200, 1.10 GHz, up to 2.50 GHz with turbo boost technology
	System Memory	4GB SODIMM DDR3L-1600 (up to 8GB)
	Storage	64GB solid state drive (SSD), optional up to 512GB
	OS	Windows 10 IoT Enterprise (64 bit)
Wireless Communications	WLAN	802.11 a/b/g/n/ac
	Bluetooth	Bluetooth 5.0
	GPS	GPS, GLONASS
	WLAN	Optional 4G/LTE
Interface	Docking Connector	1 x 19-pin docking connector
	HDMI	1 x Optional micro HDMI
	USB	1 x USB3.0 (Type A), 1 x USB3.0 (Type C) Note: The USB Type-C connector follows USB 3.0 standard, which does not support Alternate Mode (ALT) or USB Power Delivery (USB-PD).
	Audio Connector	1 x Audio Combo connector (Mic in or Line Out)
	Power Input	1 x Power Jack
Audio	Microphone	Built-in dual Digital Mic with Noise Cancellation
	Speaker	2 x 1 watt speaker
Camera	Front Camera	2 MP webcam
	Rear Camera	8 MP rear camera with autofocus and LED light
Data Capture	Barcode	Optional 1D/2D Barcode Reader
	RFID	Optional HF RFID reader 13.56 MHz
Security Function	Security	<ul style="list-style-type: none"> • Password security for user and hard disk lock • Trusted Platform Module (TPM) 2.0
	MDM	SOTI Mobicontrol compatible
Keyboard and Input	Physical Buttons	On-screen QWERTY keyboard Button 1 x power, 1 x Home, 2 x volume key, 2 x function key (Programmable configured by Hottab Utility)
	Touch	• 10-point multi touch, support Rain, Glove, Stylus
	LED Indicator	Power, Battery, HDD, RF
Power Management	Power Input	12-19V DC
	Battery	7.4V, typ. 5140 mAh Li-Polymer Battery (2S1P) 7.4V, typ. 10280 mAh Li-Polymer Battery (2S2P, optional)
	Battery Operating Time	Std. Battery: 8 hours High Capacity: 16 hours
	AC Adapter	100-240V, 50-60Hz / 19V DC
Mechanical Specification	Dimensions	271.8 x 197.2 x 19 mm (10.7 x 7.76 x 0.75 inches)
	Weight	1.2 kg (2.7 lbs) with standard battery, 1.4 kg (3.1 lbs) with optional high capacity battery
	Housing	Rugged housing enhanced with antimicrobial properties
Environment	Operating Temp.	0°C to 35°C
	Storage Temp.	-30°C to 70°C
	Humidity	10% to 90% RH, non-condensing
	Shock	MIL-STD-810G Method 516.6 Procedure I
	Vibration	MIL-STD-810G Method 514.6 Procedure I
	Drop	MIL-STD-810G Method 516.6, 4 ft to concrete
Certificates	IP Rating	IP65 certified, dustproof and waterproof
	EMC & Safety	<ul style="list-style-type: none"> • CE, FCC, IC • IEC 60601-1, IEC 60601-1-2, EN 60601-1, ANSI/AAMI ES 60601-1

Appendix C: Order Information

M101P-ME healthcare tablet available for order in the following configurations:

Order Information			
	4G / LTE	1D/2D Barcode Reader	HF RFID
M101P-ME	Optional	N/A	N/A
M101P-MEHF	Optional	N/A	Default
M101P-MEBH	Optional	Default	N/A
M101P-MEH2	Optional	Default	Default

Appendix C: Meaning of symbols on the unit

Symbol	Description	Symbol	Description	Symbol	Description
	This device complies with the Healthcare Device Directive 93/42/EEC		Indicates this device is compliance with Part 15 of FCC rules (Class A)		Serial number
	This way up		Stacking limit by number		Direct current
	Fragile		Weight		Keep away from rain
	Temperature limits		Recycling symbol for corrugated cardboard used for packaging		Consult operating instruction for use.
	TUV SUD Mark		Indicates device is approved according To The UL DEMKO Regulation		General warning sign
	Indicates device is approved according To The VCCI Regulation		MEDICAL - GENERAL MEDICAL EQUIPMENT AS TO ELECTRICAL SHOCK, FIRE AND MECHANICAL HAZARDS ONLY IN ACCORDANCE WITH ANSI/AAMI ES 60601-1 (2005) + AMD (2012) and CAN/CSA-C22.2 No.6060-1 (2014)		UL MED mark

Winmate Inc.
9F, No.111-6, Shing-De Rd., San-Chung District,
New Taipei City 24158, Taiwan, R.O.C
www.winmate.com