

# 15.6" Healthcare Display

## W15L100-PTA3-ME



CE FC 

# User Manual

---

# Contents

<b>Chapter 1: Read Me First</b>	<b>3</b>
1.1 Advisory Conventions	4
1.2 Warning and Cautions	5
<b>Chapter 2: Introduction</b>	<b>7</b>
2.1 Unpacking	8
2.2 Description of Parts	9
2.3 Physical Buttons	10
<b>Chapter 3: Installation</b>	<b>11</b>
3.1 VESA Mount	12
3.2 Connecting Peripherals	13
<b>Chapter 4: Operating the Device</b>	<b>14</b>
4.1 Turning On and Off	15
4.2 Frequency Table	16
4.3 Troubleshooting	17
<b>Chapter 4: Operating the Device</b>	<b>18</b>
4.1 Important Information	19
<b>Appendix</b>	<b>29</b>
Appendix A: Technical Specifications	30

## **Chapter 1: Read Me First**

This chapter gives you product overview, describes features and hardware specification. You will find all accessories that come with the device in the packing list. Mechanical dimensions and drawings included in this chapter.

---

## 1.1 Advisory Conventions

Four types of advisories are used throughout the user manual to provide helpful information or to alert you to the potential for hardware damage or personal injury. These are Notes, Important, Cautions, and Warnings. The following is an example of each type of advisory.

**Note:**

A note is used to emphasize helpful information

**Important:**

An important note indicates information that is important for you to know.

**Caution**

A Caution alert indicates potential damage to hardware and explains how to avoid the potential problem.

**Warning!**

An Electrical Shock Warning indicates the potential harm from electrical hazards and how to avoid the potential problem.

## 1.2 Warning and Cautions



**Warning!**

The apparatus shall not be exposed to dripping or splashing. No objects filled with liquids, such as vases, shall be placed on the apparatus.



**Warning!**

To prevent injuries, firmly fix the unit to the floor or wall following the installation manual.



**Warning!**

If the W15L100-PTA3 should be used adjacent to or stacked with other equipment, it should be observed to verify normal operation in the configuration in which it will be used.



**Caution**

When you dispose of the unit or accessories, you must obey the laws in the relative area or country and the regulations in the relative hospital regarding environmental pollution.



**Caution**

When installing, the installation space must be secured in consideration of the ventilation and service operation. Leave a space 4 cm (1 5/8 inches) or more behind, 10 cm (4 inches) or more from the left and right sides of, 6 cm (2 3/8 inches) or more from the bottom side of, and 30 cm (11 7/8 inches) or more above the unit.

### **Warning on power connection:**



**Warning!**

Use a proper power cord for your local power supply. Use the approved Power Cord (3-core mains lead) / Appliance Connector / Plug with earthing-contacts that conforms to the safety regulations of each country if applicable. Use the Power Cord (3-core mains lead) / Appliance Connector / Plug conforming to the proper ratings (Voltage, Ampere). If you have questions on the use of the above Power Cord / Appliance Connector / Plug, please consult qualified service personnel.

**For the customers in U.S.A. and Canada**

Please use the following power supply cord.

<b>United States and Canada</b>	
Plug Type	HOSPITAL GRADE*
Cord Type	Min. Type SJT Min. 18 AWG
Minimum Rating for Plug and Appliance Couplers	10A/125V
Safety Approval	UL Listed and CSA



**Note:** Grounding reliability can only be achieved when the equipment is connected to an equivalent receptacle marked "Hospital Only" or "Hospital Grade".

**Caution**

This unit is heavy. Make sure to unpack and move the unit with two or more people.

## **Chapter 2: Introduction**

This chapter gives you product overview, describes features and hardware specification. You will find all accessories that come with the device in the packing list. Mechanical dimensions and drawings included in this chapter.

---


## 2.1 Unpacking

### Owner's Record

The model and serial numbers are located at the rear of the unit. Record these numbers in the spaces provided below. Refer to these numbers whenever you call upon your Winmate dealer regarding this product.

Model No. \_\_\_\_\_ Serial No. \_\_\_\_\_


Check and identify the supplied accessories:

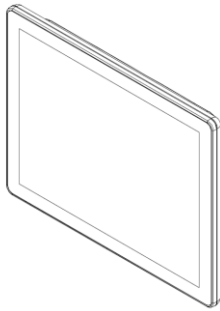

- Display




- User Manual (Hardcopy)

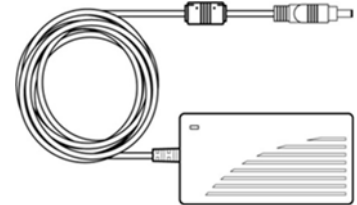


- AC Adapter




- Power Cord

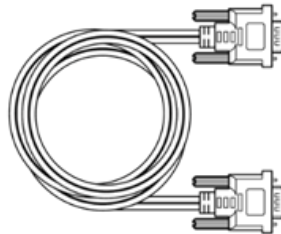


- VGA Cable

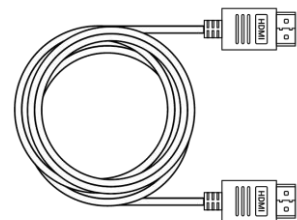


- HDMI Cable

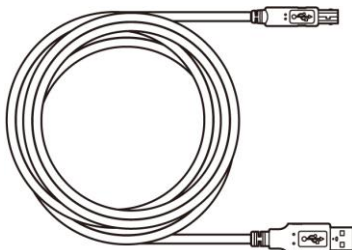

- USB Cable for Touch

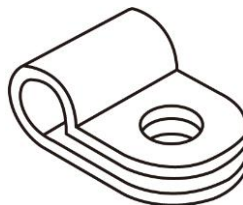

- Cable Clamp x 3 pcs (Optional)

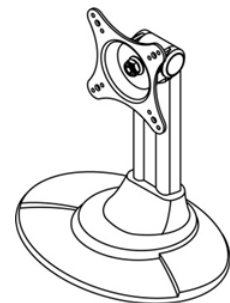


- VESA Desk Stand (Optional)





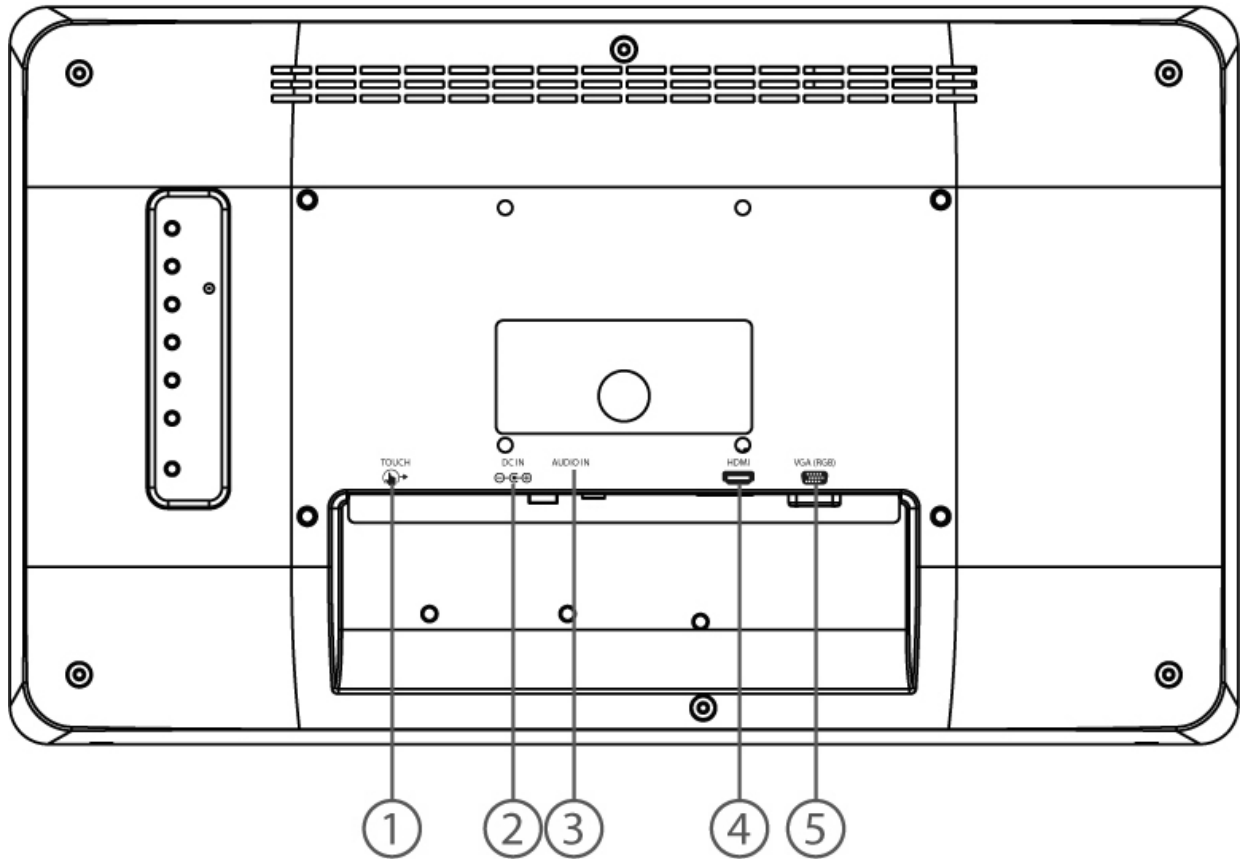


## 2.2 Description of Parts

### Rear View




### Connectors








1	TOUCH 	<b>Touch</b> - Connects USB for touch capabilities.
2	DC IN 	<b>DC IN</b> – Provides power for the monitor, accepts 12V DC. <i>Example: AC cord or DC power supply. If both are provided, choose only one.</i>
3	AUDIO IN 	<b>Audio In (Optional)</b> – Transmits audio input signal. <i>Example: Computer audio to be played through the monitor speakers.</i>
4	HDMI 	<b>HDMI</b> - Transmits and protects copyrighted digital video and audio. <i>Example: An HD tuner to an HD ready TV.</i>
5	VGA (RGB) 	<b>VGA (RGB)</b> - Transmits video from a PC to a monitor. <i>Example: A notebook PC to a monitor.</i>
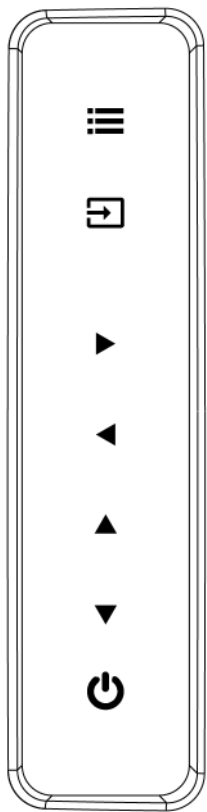
## 2.3 Physical Buttons

Physical buttons (OSD Control Panel) are located on the rear side of the Display.

### Physical Buttons



	<b>Menu</b> – Press to enable the main menu.
	<b>Input</b> – Press to ENTER/ Change video source.
	<b>Right</b> – Press to move right in the navigation menu/ Hot Key manual enable.
	<b>Left</b> – Press to move left in the navigation menu/ Hot Key information.
	<b>Up</b> – Press to Auto adjust settings.
	<b>Down</b> – Press to exit/ Auto color.
	<b>Power</b> – Press to power on or power off the device.

## **Chapter 3: Installation**

This chapter describes how to mount the display and tells you important information on power supply, adapter and precautions tips. Pay attention to power considerations.

---


### 3.1 VESA Mount

This device supports VESA mount and provides various types of mounting options to fit any industrial use.

Size	VESA Plate	Screw Size
15.6"	75x75 mm, 200x100 mm	M4 x 5

#### Mounting Instruction

Use Philips M4x5 screws to fix the desk stand to VESA holes on the back cover of the device.

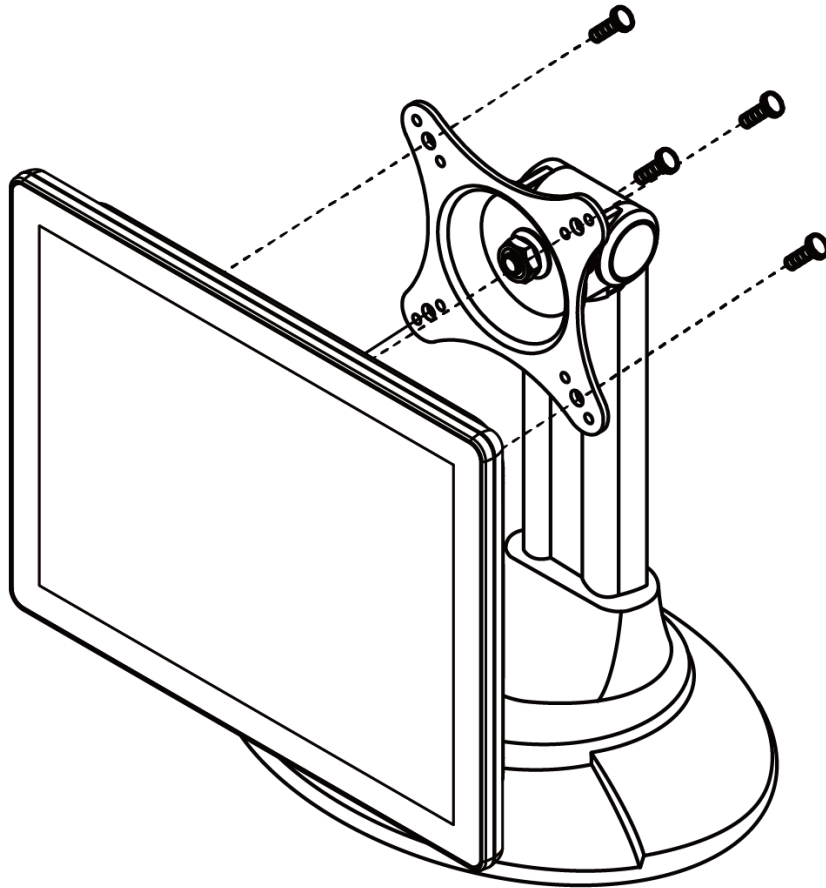

*\*The picture is for demonstration purposes only. VESA Mount accessories are not supplied by Winmate.*


## 3.2 Connecting Peripherals

Use VGA or HDMI cable to connect your display to external device. Connect USB cable for touch capabilities. Use cable clamps (optional accessory) to fix the position of cables.



**Note:**

Please adjust screen resolution to the native resolution.  
Recommended resolution 1920 x 1080.

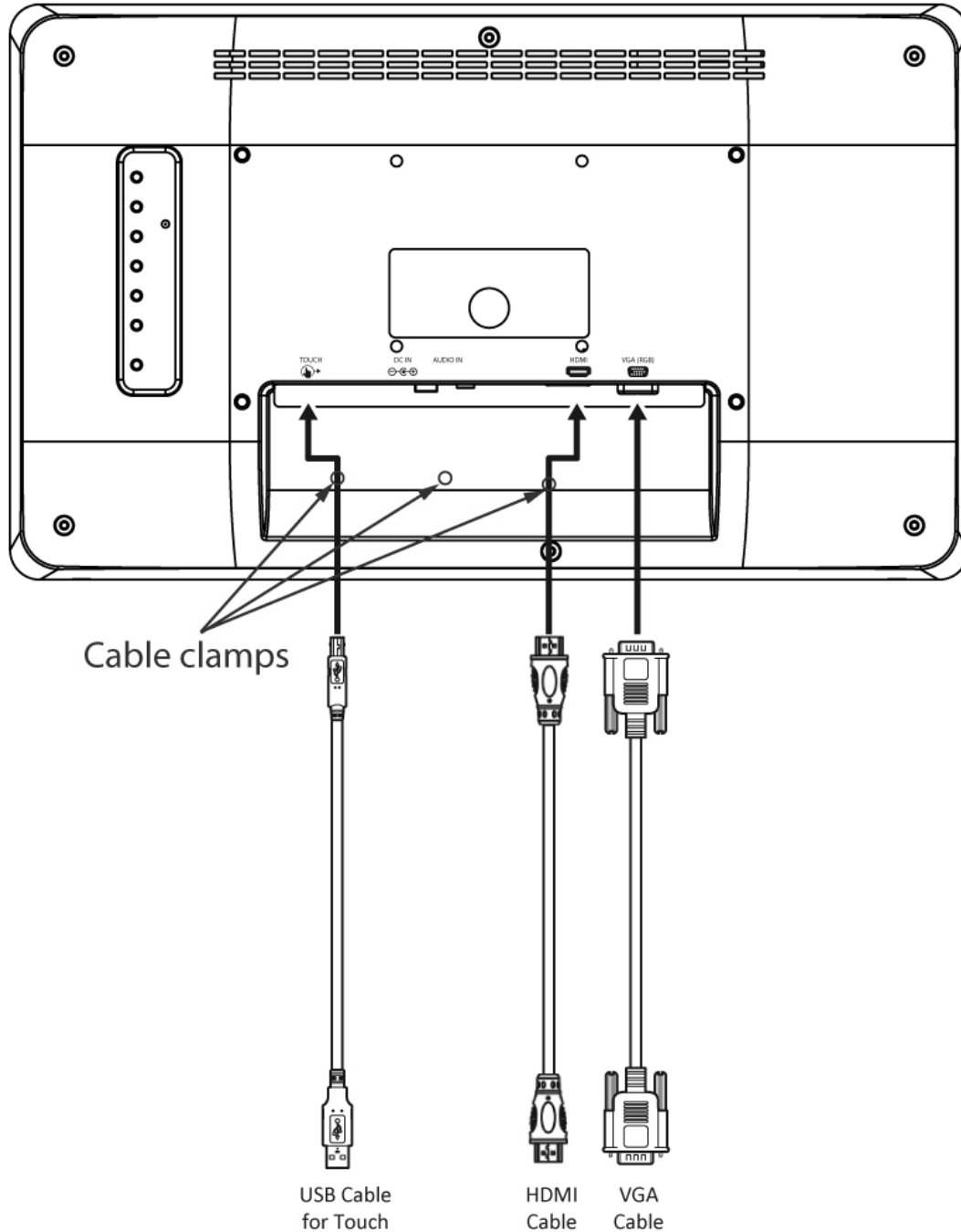

## **Chapter 4: Operating the Device**

This chapter tells you important information on power supply, adapter and precautions tips. Pay attention to power considerations.



## 4.1 Turning On and Off


Follow the following steps to turn on your device:

1. Connect the AC adapter (12V/36W) to the DC power connector located on the back side of the display.
2. Connect the power cord to AC adapter.
3. Plug the power cord to the AC outlet.
4. Press **Power** button on the rear OSD panel.



**WARNING!**

To avoid risk of electric shock, this equipment must only be connected to a supply main with protective earth.

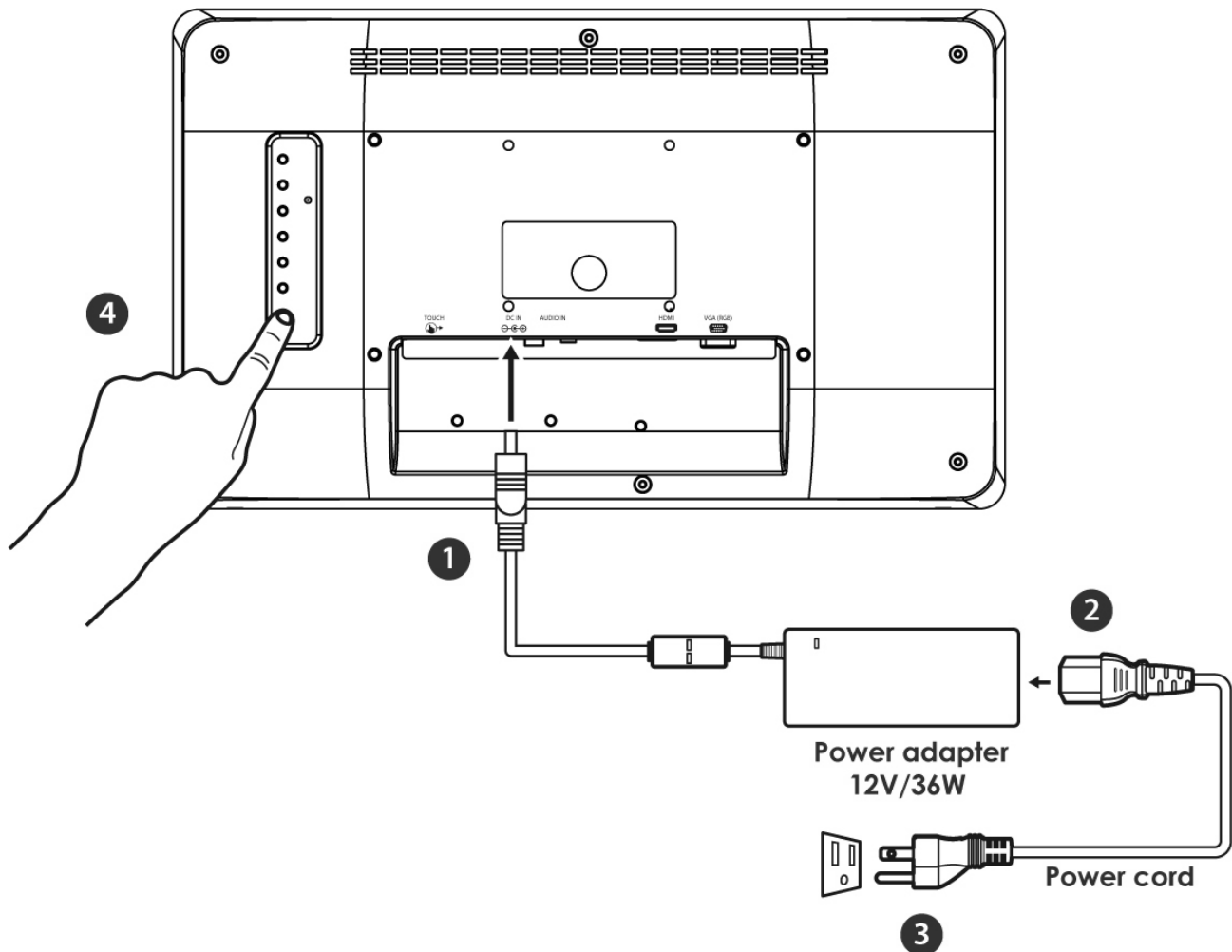

To turn off the device, please disconnect the power cord from an electric outlet.



**Caution**

To fully disengage the power, please disconnect the input plug from the ac outlet.

## 4.2 Frequency Table

Signal Name	Vertical Frequency(Hz)	VGA	HDMI
640 x 480	60	✓	✓
	72	✓	✓
	75	✓	✓
480P	60	✓	✓
	72	✓	✓
	75	✓	✓
800 x 600	60	✓	✓
	72	✓	✓
	75	✓	✓
1024 x 768	60	✓	✓
	72	✓	✓
	75	✓	✓
720P	60	✓	✓
	72	✓	✓
	75	✓	✓
1280 x 1024	60	✓	✓
	72	✓	✓
	75	✓	✓
1600 x 1200	60	✓	✓
	72	✓	✓
	75	✓	✓
1920 x 1080	60	✓	✓
	72	✓	✓
	75	✓	✓



## 4.3 Troubleshooting

Problem	Check the following
<b>There is a black dot or dead pixel dot on the screen</b>	A missing pixel does not constitute an out of spec. defective product
<b>Cannot turn power on</b>	Turn off the power supply, and check that the AC cord or DC cord are securely inserted. After checking, turn on the power supply again.
<b>It takes time for image to appear</b>	Since the monitor processes various signals digitally to reproduce beautiful images, it may take time before images appear when it is turned on, input is switched, or when the mode is switched to "Multi Window".
<b>There are spots on the screen</b>	There may be electrical distortion from vehicles, trains, high voltage lines or fluorescent lamps.
<b>Bad color</b>	Color density or tint control may not be adjusted properly. (See the adjusted value of image.)
<b>Image appears and disappears</b>	Input may not be connected properly.
<b>When changing the scan size, the top and bottom part of the screen does not show images</b>	When using a video software program (such as a cinema size program) with a screen wider than one in the 16:9 mode, blank areas separate from the images are formed at the top and bottom of the screen.
<b>Image contour flickers</b>	Due to the characteristics of the display control, contour of animated parts of images may seem to flicker, but that is not a breakdown.
<b>Color is extremely bad</b>	Input configuration "RGB"/"YPBPR" may not be selected properly.
<b>The Power indicator is flashing orange</b>	After checking the number of flashes, turn off the power and contact the dealer where the monitor was purchased.

## **Chapter 4: Operating the Device**

This chapter includes important information to consider when operating the device.



## 4.1 Important Information

### Indications for Use / Intended Use

#### Product Name

LCD Monitor

#### Definition

An electrically-powered LCD Monitor designed as an output device for displaying data (e.g., patient images, videos) from endoscopic/ laparoscopic camera systems and other compatible healthcare imaging systems. It is typically a mounted liquid crystal display (LCD) or light-emitting diode (LED) unit, and may include an antimicrobial material to prevent cross-contamination in healthcare environment. The Monitor is a widescreen, high-definition, healthcare grade for use during minimally invasive surgical procedures and is suitable for hospital operating rooms, surgical centers, clinics, doctors' offices and similar healthcare applications.

#### Classification information

- Power by Class I power supply
- No protection against ingress of water (IPX0)
- No Applied part
- Mode of operation: Continuous operation
- Provides isolation: MOPP

### **Customers in the U.S.A**

This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment uses and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications.

All interface cables used to connect peripherals must be shielded to comply with the limits for a digital device pursuant to Subpart B of part 15 of FCC Rules.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

### **Customers outside the U.S.A**

This product has been manufactured by Winmate Inc.

Inquiries related to product compliance based on European Union legislation shall be addressed to the authorized representative, Winmate. For any service or guarantee matters, please refer to the addresses provided in the separate service or guarantee documents.

**Important safeguards/notices for use in the healthcare applications**

1. All the equipment connected to this unit shall be certified per Standard IEC60601-1, IEC60950-1, IEC60065 or other IEC/ISO Standards applicable to the equipment.
2. Furthermore, all configurations shall comply with the system standard IEC60601-1-1. Everybody who connects additional equipment to the signal input part or signal output part configures a healthcare system, and is therefore, responsible that the system complies with the requirements of the system standard IEC60601-1-1.
3. If in doubt, consult the qualified service personnel.
4. The leakage current could increase when connected to other equipment.
5. For this equipment, all accessory equipment connected as noted above, must be connected to mains via an additional isolation transformer conforming to the construction requirements of IEC60601-1 and providing at least basic insulation.
6. This equipment generates, uses, and can radiate radio frequency energy. If it is not installed and used in accordance with the instruction manual, it may cause interference to other equipment. If this unit causes interference (which can be determined by unplugging the power cord from the unit), try these measures: Relocate the unit with respect to the susceptible equipment. Plug this unit and the susceptible equipment into different branch circuit.

**Important EMC notices for use in the healthcare applications**

The display needs special precautions regarding EMC and needs to be installed and put into service per the EMC information provided in the instructions for use.

The portable and mobile RF communications equipment such as cellular phones can affect the display.

**Warning!**

The use of accessories and cables other than those specified, with the exception of replacement parts sold by Winmate Inc., may result in increased emissions or decreased immunity of the device.

### Guidance and manufacturer's declaration-electromagnetic emissions

The W15L100-PTA3-ME is intended for use in the electromagnetic environment specified below. The customer or the user of the W15L100-PTA3-ME should assure that it is used in such an environment.

Emission test	Compliance	Electromagnetic environment-guidance
RF emissions CISPR 11	Group 1	The W15L100-PTA3-ME uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The W15L100-PTA3-ME is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	D	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

### Guidance and manufacturer's declaration - electromagnetic immunity

The W15L100-PTA3-ME is intended for use in the electromagnetic environment specified below. The customer or the user of the M270TF-XXX / M320TF-XXX should assure that it is used in such as environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±15 kV air	±8 kV contact ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/ output lines	±2 kV for power supply lines ±1 kV for input/ output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV line(s) to line(s) ±2 kV line(s) to earth	±1 kV differential mode ±2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 5% (> 95% dip in) for 0.5 cycle 40% (60% dip in) for 5 cycles 70% (30% dip in) for 25 cycles < 5% (> 95% dip in) for 5 sec	< 5% (> 95% dip in) for 0.5 cycle 40% (60% dip in) for 5 cycles 70% (30% dip in) for 25 cycles < 5% (> 95% dip in) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the W15L100-PTA3-ME requires continued operation during power mains interruptions, it is recommended that the W15L100-PTA3-ME powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital

NOTE: is the A.C. mains voltage prior to application of the test level.

**Guidance and manufacturer's declaration - electromagnetic immunity**

The W15L100-PTA3-ME is intended for use in the electromagnetic environment specified below. The customer or the user of the W15L100-PTA3-ME should assure that it is used in such as environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms 150 kHz to 80 MHz	<p>Portable and mobile RF communications equipment should be used no closer to any part of the W15L100-PTA3-ME, including cables, than the recommended separation distance calculated from the equation appliance to the frequency of the transmitter.</p> <p>Recommended separation distance: <math>d</math></p> $d = \left[ \frac{3,5}{v_1} \right] \sqrt{p}$
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m 80 MHz to 2.5 GHz	<p><b>Recommended Separation Distance</b></p> $d = \left[ \frac{3,5}{E_1} \right] \sqrt{p} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \left[ \frac{7}{E_1} \right] \sqrt{p} \quad 80 \text{ MHz to } 2.5 \text{ GHz}$ <p>Where <math>P</math> is the maximum output power rating of the transmitter in watts (W) per the transmitter manufacturer and <math>d</math> is the recommended separation distance in meters (m).                      Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, <sup>A</sup> should be less than the compliance level in each frequency range.  <sup>B</sup> Interference may occur near equipment marked with following symbol:</p>

NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the W15L100-PTA3-ME is used exceeds the applicable RF compliance level above, the W15L100-PTA3-ME should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the W15L100-PTA3-ME.

### Recommended separation distances between portable and mobile RF communications equipment and the W15L100-PTA3-ME

The W15L100-PTA3-ME is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the W15L100-PTA3-ME can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the W15L100-PTA3-ME as recommended below, per the maximum output power of the communications equipment.

Rated maximum output power [W] of transmitter	Separation distance per frequency of transmitter [m]		
	150 kHz to 80 MHz $d = 1.2$	80 MHz to 800 MHz $d = 1.2$	800 MHz to 2.5 GHz $d = 2.3$
	$d = \left[ \frac{3,5}{v_1} \right] \sqrt{p}$	$d = \left[ \frac{3,5}{E_1} \right] \sqrt{p}$	$d = \left[ \frac{7}{E_1} \right] \sqrt{p}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated a maximum output power not listed above, the recommended separation distance  $d$  in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) per the transmitter manufacturer.

NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

### General Safety

- W15L100-PTA3-ME is a DC powered device. Use with the supplied AC adaptor. Operate the unit on 100-240V AC only.
- For plug-in equipment, the power outlet socket must be located near the equipment and must be easily accessible.
- Use a power cord that matches the voltage of the power outlet, which has been approved and complies with the safety standard of your particular country.
- The nameplate indicating operating voltage, etc. is located on the AC adaptor.
- Should any solid object or liquid fall into the cabinet, unplug the unit and have it checked by qualified personnel before operating it any further.
- Unplug the unit from the wall outlet if it is not to be used for several days or more.
- To disconnect the AC power cord, pull it out by grasping the plug. Never pull the cord itself.
- The socket-outlet shall be installed near the equipment and shall be easily accessible.

### Healthcare use

- Please do not touch patient and this healthcare device at the same time.
- Users must not allow SIP/ SOPs and the patient to come into contact at the same time.
- Grounding reliability can only be achieved when the equipment is connected to an equivalent receptacle marked "Hospital Only" or "Hospital Grade".

### Installation

- This device can only be installed by manufacturer authorized and trained personnel.
- Prevent internal heat build-up allowing adequate air circulation.
- Do not place the unit on surfaces (rugs, blankets, etc.) or near materials (curtains, draperies) that may block the ventilation holes.
- Do not install the unit near heat sources such as radiators or air ducts, or in a place subject to direct sunlight, excessive dust, mechanical vibration or shock.
- Do not place the monitor near equipment which generates magnetism, such as a transformer or high voltage power lines.

- Use suitable mounting apparatus to avoid risk of injury.

### **Precautions for connecting this unit with other healthcare devices**

Before you utilize this device and/or connect this device to any other healthcare device, please be aware of and abide by the following precautions:

- Before actually using this device for healthcare practice, please check and confirm that you do not experience any discomfort in the use of this monitor
- If you experience or are likely to experience discomfort, please refrain from using this device.
- Generally, discomfort (such as eye strain, fatigue, nausea, or motion sickness) can be provoked by quick movements of video picture, focal positioning of video images, distances between moving objects and changing image colors.
- Before prolonged use, make sure the image of the connected healthcare device is displayed properly.

### **Use with an electrosurgical knife, etc.**

If this unit is used together with an electrosurgical knife, etc., the picture may be disturbed, warped or otherwise abnormal because of strong radio waves or voltages from the device. This is not a malfunction. When you use this unit simultaneously with a device from which strong radio waves or voltages are emitted, confirm the effect of this before using such devices, and install this unit in a way that minimizes the effect of radio wave interference.

### **Precautions for using this unit safely**

- Some people may experience discomfort (such as eye strain, fatigue, or nausea) while watching video images. It is recommended that all viewers take regular breaks while watching video images. The length and frequency of necessary breaks will vary from person to person. You must decide what works best.
- Avoid watching the display in environments where your head may shake, because there is a higher possibility that you experience discomfort.

### **Precautions modifications**

- No modification of this equipment is allowed.
- Do not modify this equipment without authorization of the manufacturer
- If this equipment is modified, appropriate inspection and testing must be conducted to ensure continued safe use of the equipment.
- If there is any other problem, please consult or return the device to your distributor. Do not try to repair a defective device.



**Recommendation to use more than one unit**

As problems, can occasionally occur, when the monitor is used under critical conditions, we strongly recommend you use more than one unit or prepare a spare unit for replacement.

**About the LCD Display Panel**

- The LCD panel fitted to this unit is manufactured with high precision technology, giving a functioning pixel ratio of at least 99.99%. Thus, a very small proportion of pixels may be “stuck”, either always off (black), always on (red, green, or blue), or flashing. In addition, over a long period of use, because of the physical characteristics of the liquid crystal display, such “stuck” pixels may appear spontaneously. These problems are not a malfunction.
- Do not leave the LCD screen facing the sun as it can damage the LCD screen. Take care when you place the unit by a window.
- Do not store the monitor FACE DOWN.
- Do not push or scratch the LCD screen.
- Do not place a heavy object on the LCD screen. This may cause the screen to lose uniformity.
- If the unit is used in a cold place, a residual image may appear on the screen. This is not a malfunction. When the monitor becomes warm, the screen returns to normal.
- The screen and the cabinet become warm during operation. This is not a malfunction.

**Images that may cause burn-in**

- Masked / boarded images with aspect ratios other than 16:9
- Color bars or images that remain static for a long time
- Continuous characters or messages displaying on the screen

**To reduce the risk of burn-in**

Turn off the character displays from connected equipment.

**About the screen protect panel**

The screen protect panel is made of toughened glass, but there is a possibility that it may crack.

- Handle with care. Avoid strong impact, such as dropping from a high place or an object swinging into it.
- Do not scratch the panel with a sharp object or place it in harm’s way.

**A long period of use**

- Due to the characteristics of LCD panel, displaying static images for extended periods, or using the unit repeatedly in a high temperature/high humidity environments may cause image smearing, burn-in, areas of which brightness is permanently changed, lines, or a decrease in overall brightness.
- In particular, continued displaying of an image smaller than the monitor screen, such as in a different aspect ratio, may shorten the life of the unit.
- Avoid displaying a still image for an extended period, or using the unit repeatedly in a high temperature/high humidity environment such as an airtight room, or around the outlet of an air conditioner.

**Moisture condensation**

If the unit is suddenly taken from a cold to a warm location, or if ambient temperature suddenly rises, moisture may form on the outer surface of the unit and/or inside of the unit. This is known as condensation. If condensation occurs, turn off the unit and wait until the condensation clears before operating the unit. Operating the unit while condensation is present may damage the unit.

**Cleaning before cleaning**

Be sure to disconnect the AC power cord from the AC outlet.


**Cleaning the monitor**

A material that withstands disinfection is used for the front protection plate of the healthcare use LCD monitor. The protection plate surface is specially treated to reduce reflection of light. When solvents such as benzene or thinner, or acid, alkaline or abrasive detergent or chemical cleaning cloth are used for the protection plate surface/monitor surface, the performance of the monitor may be impaired or the finish of the surface may be damaged. Take care with respect to the following:

- Clean the protection plate surface/monitor surface with a 50 to 70 v/v% concentration of isopropyl alcohol or a 76.9 to 81.4 v/v% concentration of ethanol using a swab method. Wipe the protection plate surface gently (wipe using less than 1 N force).
- Stubborn stains may be removed with a soft cloth such as a cleaning cloth lightly dampened with mild detergent solution using a swab method and then clean using the above chemical solution.
- Never use solvents such as benzene or thinner, or acid, alkaline or abrasive detergent, or chemical cleaning cloth for cleaning or disinfection, as they will damage the protection plate surface/monitor surface.
- Do not use unnecessary force to rub the protection plate surface/monitor surface with a stained cloth. The protection plate surface/monitor surface may be scratched.
- Do not keep the protection plate surface/monitor surface in contact with a rubber or vinyl resin product for a long period of time. The finish of the surface may deteriorate.

**Flat surface for better maintenance**

The design allows the user to easily wipe liquids and gel off the LCD panel and control buttons – ensuring a high standard of disinfection and cleanliness.

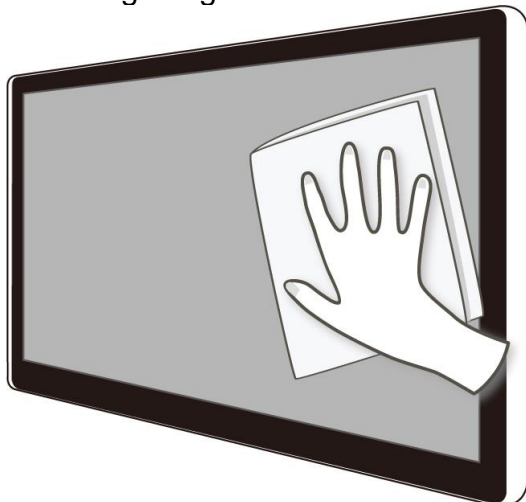**Repacking**

Do not throw away the carton and packing materials. They can be used again to repack monitor.

If you have any questions about this unit, contact your authorized dealer.

**Disposal of the unit**

Do not dispose of the unit with general waste. Do not include the monitor with household waste.

**Biological Hazard and Returns**

The structure and the specifications of this device as well as the materials used for manufacturing makes it easy to wipe and clean and therefore suitable to be used for various applications in hospitals and other healthcare environments, where procedures for frequent cleaning are specified.
























However, normal use shall exclude biological contaminated environments, to prevent spreading of infections.

Therefore use of this device in such environments is at the exclusive risk of Customer. In case this device is used where potential biological contamination cannot be excluded.

Customer shall implement the decontamination process as defined in the latest edition of the ANSI/AAMI ST35 standard on each single failed Product that is returned for servicing, repair, reworking or failure investigation to Seller (or to the Authorized Service Provider). At least one adhesive yellow label shall be attached on the top site of the package of returned Product and accompanied by a declaration statement proving the Product has been successfully decontaminated.

Returned Products that is not provided with such external decontamination label, and/or whenever such declaration is missing, can be rejected by Seller (or by the Authorized Service Provider) and shipped back at Customer expenses.

## Meaning of symbols on the unit

Symbol	Description	Symbol	Description	Symbol	Description
	Potential equalization terminal (Sets other devices to same potential level).		This device complies with the Healthcare Device Directive 93/42/EEC		Serial number
	Date of manufacture		Manufacturer		Direct current
	This way up		Stacking limit by number		Keep away from rain
	Fragile		Weight		Consult operating instruction for use.
	Temperature limits		Recycling symbol for corrugated cardboard used for packaging		General warning sign
	Indicates user need to read user manual before using the device		UL Listing Mark		UL Recognized Component Mark
	TUV SUD Mark		Indicates device is approved according To The UL DEMKO Regulation		Indicates device is approved according To The VCCI Regulation
	Indicates this device is compliance with Part 15 of FCC rules (Class A)		Indicates this device must not throw in the trash		

### For business users in the European Union

If you wish to discard electrical and electronic equipment, please contact your dealer or supplier for further information. Information on Disposal in other

### Countries outside the European Union.

Winmate reserves the right to make changes in specifications and features shown herein, or discontinue the product at any time without notice or obligation."



Information on Disposal for Users of Waste Electrical & Electronic Equipment (private households)

This symbol on the products and/or accompanying documents means that used electrical and electronic products should not be mixed with general household waste.

For proper treatment, recovery and recycling, please take these products to designated collection points, where they will be accepted on a free of charge basis. Alternatively, in some countries you may be able to return your products to your local retailer upon the purchase of an equivalent new product. Disposing of this product correctly will help to save valuable resources and prevent any potential negative effects on human health and the environment which could otherwise arise from inappropriate waste handling. Please contact your local authority for further details of your nearest designated collection point.

Penalties may be applicable for incorrect disposal of this waste, in accordance with national legislation.

## **Appendix**

This section tells you additional information about the product.



## Appendix A: Technical Specifications

Model Name	W15L100-PTC3-ME	
Display	Size/Type	15.6" TFT
	Resolution	1920 x 1080
	Brightness	300 <i>cd/m<sup>2</sup></i>
	Contrast Ratio	700:1 (typ.)
	Viewing Angle	~85~85 (H); ~85~85 (V)
	Max Colors	16.2M
	Touch	Projected Capacitive Touch
Input/ Output	Input Ports	1 x VGA 1 x HDMI 1 x Audio 1 x USB for Touch
	OSD Membrane	7 x Control buttons on the rear side
	Speaker	2 x 1W Speaker
Power Specifications	Power Input	12V DC 2.5 mm, with External AC to DC Adapter (comply with Level VI efficiency standard)
	Power Consumption	10W typical, normal operation
Mechanical Specification	Mounting	VESA Mount (75 x 75 mm, 100 x 200 mm), VESA Screws M4x7mm, flat head
	Dimensions (W x H x D)	387.8 x 232.9 x 38.45 (mm)
	Housing	Back cover white color, plastic casing
	OSD Buttons	Menu, Input, Right, Left, Up, Down, Power
Environment Considerations	Operating Temperature	0°C to 35°C
	Storage Temperature	-20°C to 60°C
	Humidity	10%~90% (non-condensing)
	LCD Panel Lifetime	50, 000 hours
	Testing and Certifications	<ul style="list-style-type: none"> <li>• IEC 60601-1:2005 + A1:2015 (Ed. 3.1)</li> <li>• IEC 60601-1-6:2010 + A1:2013 (Ed. 3.1)</li> <li>• IEC 62366:2007 + A1:2014 (Ed. 1.1)</li> <li>• ANSI/AAMI ES60601-1:2005/ A1:2012and C1:2009/(R)2012and A2:2010/(R)2012</li> <li>• CAN/CSA-C22.2 No.60601-1:14</li> <li>• EN60601-1-2 (Ed. 4.0)</li> <li>• EN55032/EN55024, FCC part 15B</li> <li>• EN60950-1</li> <li>• RoHS-2, REACH, WEEE compliant</li> </ul>



---

Winmate Inc.  
9F, No.111-6, Shing-De Rd., San-Chung District,  
New Taipei City 24158, Taiwan, R.O.C  
[www.winmate.com](http://www.winmate.com)